m-prep®

DNA Sample Preparation

DNA Sample Preparation Kit for use with
Meridian Bioscience DNA Amplification Assays

REF 281110

IVD

INTENDED USE

The **M-prep** DNA Sample Preparation Kit, for use with Meridian Bioscience DNA Amplification Assays, is a size exclusion chromatography system for purification of genomic nucleic acids from biological specimens. This product is intended for use in conjunction with in vitro diagnostic medical devices.

SUMMARY AND EXPLANATION OF THE TEST

The **M-prep** Sample Preparation kit is a nucleic acid extraction and purification product used to prepare biological specimens for molecular amplification and detection. The extraction and purification process is performed by gravity flow and does not require specialized laboratory equipment.

The **M-prep** DNA Sample Preparation kit contains size exclusion columns, sample preparation buffers and screw-top microcentrifuge tubes. **M-prep** Columns provide the matrix for purification of nucleic acids while the included buffers facilitate cell lysis and sample elution. Effluent, including processed samples, is collected in screw-top **M-prep** tubes. Each polypropylene copolymer **M-prep** Column contains size-exclusion chromatography resin between two porous filters. Genomic materials to be prepared for amplification are first treated with **M-prep** Buffer I; cells are disrupted and nucleic acids are released. Sample material is applied to the **M-prep** Column and allowed to move through the column by gravity, with the largest molecules moving through the column matrix first, or fastest. **M-prep** Buffer II is applied to the **M-prep** Column, facilitating movement of the sample through the column matrix. **M-prep** Buffer II contains a non-reactive colorant that provides a visual indicator of sample progression. Sample effluent following **M-prep** Buffer II addition is discarded as waste. **M-prep** Buffer III is applied to the **M-prep** Column after Buffer II has entered the gel matrix. Sample effluent collected after the addition of **M-prep** Buffer III contains nucleic acids suitable for use in molecular amplification applications.

BIOLOGICAL PRINCIPLES

The **M-prep** DNA Sample Preparation Kit is a system that uses size exclusion chromatography for extraction and purification of genomic nucleic acids from biological specimens. The purified nucleic acids are suitable for amplification and detection in molecular-based assay platforms.

REAGENTS/MATERIALS PROVIDED

The maximum number of tests obtained from this test kit is listed on the outer box.

- M-prep Buffer I (item #2904):** Lysis solution containing 0.3% Sodium dodecyl sulfate and 0.4N sodium hydroxide, pH > 12.6.
- M-prep Buffer II (item #2906):** Solution containing phenol red and 0.09% azide as a preservative.
- M-prep Buffer III (item #2907):** Tris-buffered solution with 0.09% azide as a preservative.
- M-prep Column (item #2905):** 5.0 cm long column with twist-off tip and plug seal cap containing chromatography resin in a tris-buffered solution containing sodium azide (0.09%) as a preservative.
- M-prep Tubes (item #2342):** 2.0 mL screw-top microcentrifuge tubes.

MATERIALS PROVIDED SEPARATELY

- Meridian Bioscience Chlamydia DNA Amplification Assay, Meridian Bioscience, Inc. Catalog Number: 480450
- Meridian Bioscience Gonorrhea DNA Amplification Assay, Meridian Bioscience, Inc. Catalog Number: 480850

MATERIALS NOT PROVIDED

- Disposable latex gloves, powder free

EQUIPMENT NOT PROVIDED

Not applicable.

PRECAUTIONS

- All reagents are for in vitro diagnostic use only.
- Do not interchange reagents between lots.
- Follow Biosafety Level 2 and Good Laboratory practices during testing¹. Treat all specimens and used test materials as capable of transmitting infectious agents. Do not eat drink or smoke in areas where specimens or kit reagents are handled.
- Wear disposable gloves while collecting and handling specimens and thoroughly wash hands afterwards.
- Quality Control Programs for Molecular Testing Laboratories should be employed.¹
- Dispose of used **M-prep** Columns and **M-prep** Tubes immediately after processing to reduce the possibility of contamination of the test area.
- Any serious incident that has occurred in relation to the device should be reported to Meridian Bioscience, Inc., 3471 River Hills Drive, Cincinnati, Ohio 45244 USA or Technical Support Center 800-343-3858 and competent authority of the EU Member State in which the clinician and/or patient is established.
- IMPORTANT: See SDS for additional safety and hazard information.

HAZARD and PRECAUTIONARY STATEMENTS

Signal Word

Warning

Hazard Statements

H315 - Causes skin irritation

H319 - Causes serious eye irritation

Precautionary Statements - EU (§28, 1272/2008)

P321 - Specific treatment (see .? on this label).

P264 - Wash face, hands and any exposed skin thoroughly after handling.

P280 - Wear eye protection/ face protection.

P337 + P313 - If eye irritation persists: Get medical advice/attention.

SHELF LIFE AND STORAGE

The expiration date is indicated on the kit label. Store the kit components at the temperature indicated on the label. Do not freeze.


COLUMN PREPARATION SETUP

Visually inspect each **M-Prep** Column and ensure the column bed and filters were not disturbed during transportation. Columns with disturbed resin beds or improperly placed filters may not perform correctly.

Column Set-up Step:

1. Use 1 **M-prep** Column for each sample to be tested. Remove the top cap and the bottom twist-off tip.
2. Place the column tip into an **M-prep** tube. The **M-prep** Tube should be held at a slight angle (approximately 15°) from the **M-prep** Column.
NOTE: Do not seat the column directly in the collection tube as this may create a seal and cause improper flow through the column. Refer to **Figure 1** for proper column and collection tube placement.
3. Let the **M-prep** Column drain into the **M-prep** Tube. Drained **M-prep** Columns should be used within 1 hour.
4. Proceed with DNA sample preparation. Refer to individual Meridian Bioscience Assay Package Inserts for specimen processing and test procedures.

FIGURE 1: M-prep Column and M-prep Tube placement

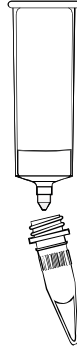LIMITATIONS

This product can only be used with Meridian Bioscience DNA Amplification Assays.

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REFERENCES

1. US Department of Health and Human Services PHS/CDC/NIH. Biosafety in microbiology and biomedical laboratories, Washington DC: US Government Printing Office, 2007.

SN11191

REV. 07/24

 Manufactured By	Meridian Bioscience, Inc. 3471 River Hills Drive Cincinnati, OHIO - 45244 USA www.meridianbioscience.com <u>Contacts:</u> Main Telephone (+1) 513.271.3700 Customer Service/Orders 800.543.1980 Technical Support Center 800.343.3858 Information Fax: 513.272.5432 Ordering Fax: 513.271.0124 E-mail: info@meridianbioscience.com
 Authorized Representative	Meridian Bioscience Europe, SRL Via Dell'Industria 7, 20035 Villa Cortese (Milano) ITALY www.meridianbioscience.com <u>Contacts:</u> Main Telephone (+39) 0331.433636 E-mail: info@meridianbioscience.eu Technical Support: MBE-TechService@meridianbioscience.eu Customer Service/Orders: • For Italian Customers: ordini@meridianbioscience.com • For Distributors / International Customers: Export.CustomerService@meridianbioscience.eu

INTERNATIONAL SYMBOL USAGE

You may see one or more of these symbols on the labeling/packaging of this product:

Key guide to symbols (Guida ai simboli, Guide des symboles, Guía de símbolos, Zeichenerklärung)

	Use By / Utilizzare entro / Utiliser jusque / Fecha de caducidad / Verwendbar bis	CONTROL +	Positive control / Controllo positivo / Contrôle positif / Control positivo / Positive Kontrolle
LOT	Batch Code / Codice del lotto / Code du lot / Código de lote / Chargenbezeichnung	CONTROL -	Negative control / Controllo negativo / Contrôle négatif / Control negativo / Negative Kontrolle
IVD	In vitro diagnostic medical device / Dispositivo medico-diagnostico in vitro / Dispositif médical de diagnostic in vitro / Dispositivo médico para diagnóstico in vitro / In-Vitro-Diagnostikum	EC REP	Authorized Representative in the European Community / Rappresentante Autorizzato nella Comunità Europea / Mandataire dans la Communauté européenne / Representante autorizado en la Comunidad Europea / Bevollmächtigter in der Europäischen Gemeinschaft
	Meridian products carrying the European Conformity (CE) mark fulfill the requirements of Directive 98/79/EC or the Regulation 2017/746 on in-vitro diagnostic medical devices / I prodotti Meridian recanti il marchio di Conformità Europea (CE) soddisfano i requisiti della Direttiva 98/79/CE o del Regolamento 2017/746 sui dispositivi medico-diagnostici in vitro / Les produits Meridian portant la marque de Conformité européenne (CE) sont conformes aux exigences de la Directive 98/79/CE ou du Règlement 2017/746 relatifs aux dispositifs médicaux de diagnostic in vitro / Los productos de Meridian que llevan la marca de conformidad europea (CE) cumplen los requisitos de la Directiva 98/79/CE o del Reglamento 2017/746 sobre los productos sanitarios para diagnóstico in vitro / Produkte von Meridian mit der CE-Kennzeichnung erfüllen die Anforderungen der EU-Richtlinie 98/79/EG bzw. der Verordnung (EU) 2017/746 über In-vitro-Diagnostika	SMP PREP DIL SPE	Sample Preparation Apparatus containing Sample Diluent / Dispositivo per la preparazione del campione contenente il diluente del campione / Système pour la préparation de l'échantillon, diluant inclus / Aparato para Preparación de Muestra con Diluyente de Muestra / System zur Probenvorbereitung, in dem sich Probenverdünnungspuffer befindet
		STERILE R	Sterilization by gamma irradiation / Sterilizzazione con raggi gamma / Stérilisation par irradiation aux rayons gamma / Esterilizado por irradiación gamma / Sterilisation durch Gammastrahlen
REF	Catalogue number / Numero di catalogo / Référence du catalogue / Número de catálogo / Bestellnummer		Do not freeze / Non congelare / Ne pas congeler / No congelar / Nicht Einfrieren
	Consult Instructions for Use / Consultare le istruzioni per l'uso / Consulter les instructions d'utilisation / Consulte las instrucciones de uso / Gebrauchsanweisung beachten	RoHS	Restriction of Hazardous Substances / Restrizione all'uso di sostanze pericolose / Limitation de substances dangereuses / Restricción de Substancias Nocivas / Beschränkung der Verwendung bestimmter gefährlicher Stoffe
	Manufacturer / Fabricante / Fabricant / Fabricante / Hersteller		Caution, consult accompanying documents / Attenzione, vedere le istruzioni per l'uso / Attention voir notice d'instructions / Atención, ver instrucciones de uso / Achtung, Begleiddokumente beachten
SN	Serial number / Numero di serie / Numéro de série / Número de serie / Seriennummer	STERILE EO	Sterilization by ethylene oxide / Sterilizzazione con ossido di etilene / Stérilisation par oxyde d'éthylène / Esterilizado por óxido de etileno / Sterilisation durch Ethylenoxid
	Contains sufficient for <n> tests / Contenuto sufficiente per "n" saggi / Contenu suffisant pour "n" tests / Contenido suficiente para <n> ensayos / Inhalt ausreichend für <n> Prüfungen	BUF RXN	Reaction Buffer / Tampone di reazione / Solution de réaction tamponnée / Tampón de Reacción / Reaktionspuffer
	Temperature limitation / Limiti di temperatura / Limites de température / Limite de temperatura / Temperaturbegrenzung		ETL Registered Mark Certified / Marchio di certificazione registrato a livello nazionale / Certifié Conforme ETL / Marca de Certificación Registrada Nacional / ETL Konform beglaubigt
	Female / Femminile / De sexe féminin / Hembra / Frau		Recycle – do not dispose of as general waste / Riciclare – non eliminare come rifiuto generico / Recycler – ne pas jeter dans une poubelle / Recicle – no desechе como basura general / Recycling- dieses Produkt nicht über den Hausmüll entsorgen
	Single Use Only / Prodotto Monouso / A usage unique / Para Un Solo Uso / nur für die einmalige Anwendung		
TEST	Test Device / Dispositivo test / Dispositif de test / Dispositivo de Prueba / Testgerät	HT TUBE	Heat Treatment Tube / Provetta per il Trattamento termico / Tube pour le traitement thermique / Tubo de tratamiento de calor / Röhrchen zur Hitzebearbeitung
	Date of manufacture / Data di fabbricazione / Date de fabrication / Fecha de fabricación / Herstellungsdatum		For IVD Performance Evaluation Only / Soltanto per valutazione delle prestazioni / Réactifs IVD réservés à l'évaluation des performances / Sólo para evaluación del funcionamiento / Nur zur IVD Leistungsbewertung
	LASER RADIATION: Avoid Exposure to Beam / RADIAZIONE LASER: Evitare l'esposizione al raggio / RAYONNEMENT LASER: Eviter toute exposition au faisceau / Radiación Laser: Evite Exposición a los Rayos / LASERSTRAHLUNG: Direkten Kontakt mit dem Strahl vermeiden		HOT SURFACE: Keep hands Away from Hot Surfaces / Superficie calda: tenere le mani lontane dalle superfici calde / SURFACES CHAUDES: Ne pas toucher les surfaces chaudes / Superficie Caliente: Mantenga las manos alejadas de la superficie caliente / Heiße Oberfläche: Kontakt mit heißen Oberflächen vermeiden
	CAUTION: Laser Radiation / ATTENZIONE: Radiazione Laser / AVERTISSEMENT: Rayonnement laser / Precaución: Radiación Laser / WARNUNG: Laserstrahlung	IPX-0	CAUTION: Protect from water / ATTENZIONE: Proteggere dall'acqua / AVERTISSEMENT: Protéger de l'humidité / Precaución: Proteja del agua / WARNUNG: Vor Feuchtigkeit schützen
	CAUTION: Risk of Danger / ATTENZIONE: Pericolo / AVERTISSEMENT: Risques de danger / Precaución: Peligroso / WARNUNG: Risikogefahr	CONTROL	Assay Control / Controllo del test / Test de contrôle / Control de Ensayo / Kontrolltest
BUF	Buffer / Soluzione tampone / Solution tamponnée / Tampón / Puffer		Warning / Avvertenze / Mise En Garde / Advertencia / Warnhinweise
MIN OIL	Mineral Oil / Olio Minerale / Huile Minérale / Aceite Mineral / Mineralöl	MEDIA	Media / Terreno di trasporto / Milieux / Medio / Medium
ST TUBE	Screw Top Tube / Provetta con tappo a vite / Tube à bouchon vissé / Tubo con tapa de rosca / Röhrchen mit Schnappverschluss	COL	Sample Preparation Column / Colonna di preparazione del campione / Colonne pour la préparation de l'échantillon / Columna de preparación de muestra / Säule zur Probenaufarbeitung
BUF SMP	Sample Buffer / Soluzione tampone per il campione / Tampon de l'échantillon / Tampón de muestra / Probenpuffer	PRE REAG	Pretreatment Reagent / Reagente di Pretrattamento / Réactif de prétraitement / Reactivo de pretratamiento / Reagenz für die Vorbehandlung
Rx Only	Prescription Use Only / Per l'uso su prescrizione medica / Uniquement sur prescription / Solo Para Uso Por Receta / verschreibungspflichtig	SMP PREP	Sample Preparation / Preparazione del campione / Préparation de l'échantillon / Preparación de Muestra / Probenvorbereitung
TUBE	Empty Tube / Provetta vuota / Tube vide / Tubo vacío / Leeres Gefäß	CH REP	Swiss Authorized Representative / Mandatario svizzero / Mandataire Suisse / Representante Autorizado Suizo / Schweizer Bevollmächtigter

For technical assistance, call Technical Support Services at 800-343-3858 between the hours of 8AM and 6PM, USA Eastern Standard Time. To place an order, call Customer Service Department at 800-543-1980.