Para-Pak® Systems
For use with SAF
(US Patent No. 5624554)

REF 300212, 330384,
910212, 990160

IVD

Rx Only

INTENDED USE

The Para-Pak SAF is a convenient system for the routine collection, transportation, preservation, and examination of stool specimens for intestinal parasites. It is designed for easy use by individuals not trained in microbiological techniques and affords an excellent means of minimizing the adverse effects of delay in specimen transportation.

SUMMARY AND EXPLANATION OF THE TEST

Diagnosis of intestinal parasitic disease is confirmed by recovery and identification of helminth eggs and larvae, or protozoan trophozoites and cysts in the clinical parasitology laboratory. Timely collection and transportation of "fresh" stool specimens to the laboratory cannot always be assured. Workload conditions and priorities in clinical laboratories frequently do not permit immediate examination of "fresh" specimens. Procedures such as incubation, refrigeration, or freezing will not guarantee the recovery of all diagnostic stages of all parasites.^{1, 2, 5, 8, 9}

In 1972, Junod described sodium acetate-acetic acid-formalin (SAF) fixative as a multipurpose fixative-preserved permitting the recovery and identification of all diagnostic stages of intestinal parasites. In a study of over 900 specimens Scholten and Yang confirmed the suitability of SAF fixative for routine use in the clinical parasitology lab as an alternative to other fixatives.^{3, 10}

Proper use of the Para-Pak SAF systems thus assures the parasitologist that diagnostic stages of intestinal parasites will be preserved if present in the fecal material.

BIOLOGICAL PRINCIPLES

SAF provides a multipurpose fixative-preserved suitable for a variety of parasitological procedures. A direct examination may be made by diluting a drop of SAF preserved suspension in either saline or iodine. SAF preserved specimens also allow permanently stained mounts and a concentration procedure for direct examination.

The incorporation of an unpreserved specimen permits amoeba culture, rearing of hookworm larvae, bacterial culture, occult blood, and stool fat examination.¹⁰

REAGENTS/MATERIALS PROVIDED

The maximum number of tests obtained from this test kit is listed on the outer box.

1. SAF fixative: Each kit consists of one vial containing 15 mL SAF fixative with a built-in collection spoon. SAF fixative vials are also available in case quantities. Simple directions for patients and nursing personnel are also provided.

Catalog# 300212

SAF Fixative (item# 9002)
Clean Vials (item# 9003)
Mayer's Albumin (item# 9007)

Catalog# 330384

SAF (item# 9002)
Clean Vials (item# 9003)
C&S (item# 9006)
Mayer's Albumin (item# 9007)

Catalog# 910212

SAF (item# 9002)

Catalog# 990160

SAF (item# 9002)

MATERIALS NOT PROVIDED

1. Ethyl acetate (suggested) or dimethyl ether (optional)
2. Physiological saline
3. Cotton tipped applicator sticks
4. Microscope slides and coverslips
5. Centrifuge
6. Microscope
7. Transfer pipettes

PRECAUTIONS

1. All reagents are for in vitro diagnostic use only.
2. Ethyl acetate and dimethyl ether are flammable. Use in a well ventilated area. Avoid open flame. Avoid contact of the solution with skin and eyes. Should contact occur, flush with running water. Avoid breathing fumes.
3. Avoid contact of SAF solution with the skin and eyes. Should contact occur, flush with running water. If irritation develops contact a physician.
4. SAF solution is poisonous. If ingested, dilute by drinking milk or water. Call the local poison center or physician immediately.
5. Due to the infectious nature of unpreserved stools, use of gloves, care, and handwashing should be employed when the specimen is obtained and manipulated.
6. Any serious incident that has occurred in relation to the device should be reported to Meridian Bioscience, Inc., 3471 River Hills Drive, Cincinnati, Ohio 45244 USA or Technical Support Center 800.343-3858.
7. IMPORTANT: See SDS for additional safety and hazard information.

HAZARD and PRECAUTIONARY STATEMENTS

	Signal Word Danger Hazard Statements H301 - Toxic if swallowed H311 - Toxic in contact with skin H317 - May cause an allergic skin reaction H341 - Suspected of causing genetic defects H350 - May cause cancer H402 - Harmful to aquatic life H330 - Fatal if inhaled H315 - Causes skin irritation H318 - Causes serious eye damage H370 - Causes damage to organs Preccautionary Statements - EU (\$28, 1272/2008) P301 + P310 - IF SWALLOWED: Immediately call a POISON CENTER or doctor/ physician P321 - Specific treatment (see supplemental first aid instructions with this material) P280 - Wear protective gloves/ protective clothing P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P280 - Wear eye protection/ face protection P321 - See SDS Section 4 or Section 11 for specific medical treatment information P201 - Obtain special instructions before use P281 - Use personal protective equipment as required P308 + P313 - IF exposed or concerned: Get medical advice/ attention P202 - Do not handle until all safety precautions have been read and understood P264 - Wash face, hands and any exposed skin thoroughly after handling P270 - Do not eat, drink or smoke when using this product P260 - Do not breathe dust/fume/gas/mist/vapors/spray P271 - Use only outdoors or in a well-ventilated area P284 - Wear respiratory protection P272 - Contaminated work clothing should not be allowed out of the workplace P307 + P311 - IF exposed: Call a POISON CENTER or doctor/ physician P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing P310 - Immediately call a POISON CENTER or doctor/ physician P302 + P352 - IF ON SKIN: Wash with plenty of soap and water P312 - Call a POISON CENTER or doctor/ physician if you feel unwell P361 - Remove/Take off immediately all contaminated clothing P332 + P313 - If skin irritation occurs: Get medical advice/ attention P363 - Wash contaminated clothing before reuse P304 + P340 - IF INHALED: Remove to fresh air and keep at rest in a position comfortable for breathing P310 - Immediately call a POISON CENTER or doctor/ physician P405 - Store locked up P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P501 - Dispose of contents/ container to an approved waste disposal plant.
  	Para-Pak 10% Formalin

 <p>Para-Pak SAF</p>	<p>Signal Word Danger</p> <p>Hazard Statements</p> <p>H301 - Toxic if swallowed H311 - Toxic in contact with skin H318 - Causes serious eye damage H341 - Suspected of causing genetic defects H350 - May cause cancer H315 - Causes skin irritation H330 - Fatal if inhaled Contains Formaldehyde</p> <p>Precautionary Statements - EU (§28, 1272/2008)</p> <p>P301 + P310 - IF SWALLOWED: Immediately call a POISON CENTER or doctor/ physician P321 - Specific treatment (see supplemental first aid instructions with this material) P321 - See SDS Section 4 or Section 11 for specific medical treatment information P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing P310 - Immediately call a POISON CENTER or doctor/ physician P281 - Use personal protective equipment as required P201 - Obtain special instructions before use P308 + P313 - IF exposed or concerned: Get medical advice/ attention P304 + P340 - IF INHALED: Remove to fresh air and keep at rest in a position comfortable for breathing P310 - Immediately call a POISON CENTER or doctor/ physician P302 + P352 - IF ON SKIN: Wash with plenty of soap and water P310 - Immediately call a POISON CENTER or doctor/ physician P362 - Take off contaminated clothing and wash before reuse P308 + P313 - IF exposed or concerned: Get medical advice/ attention P363 - Wash contaminated clothing before reuse P333 + P313 - If skin irritation or rash occurs: Get medical advice/ attention P330 - Rinse mouth P405 - Store locked up P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P501 - Dispose of contents/ container to an approved waste disposal plant. P280 - Wear protective gloves/ protective clothing/ eye protection/ face protection P202 - Do not handle until all safety precautions have been read and understood P264 - Wash face, hands and any exposed skin thoroughly after handling P270 - Do not eat, drink or smoke when using this product P260 - Do not breathe dust/fume/gas/mist/vapors/spray P271 - Use only outdoors or in a well-ventilated area P284 - Wear respiratory protection P272 - Contaminated work clothing should not be allowed out of the workplace</p>
---	--

SHELF LIFE AND STORAGE

Expiration dating for Para-Pak SAF is indicated on the outer packaging. Store at room temperature (15-30 C). Excessive heat and cold should be avoided.

SPECIMEN COLLECTION AND PREPARATION

1. The patient should be cautioned against the use of antacids, barium, bismuth, antidiarrheal medication, or oily laxatives prior to collection of the specimen.
2. To assure recovery of parasitic elements which are passed intermittently and in fluctuating numbers, three specimens spaced a few days apart must be examined. In the case of hospitalized patients it is suggested that all fecal passages be collected for a designated length of time to avoid prolonging the hospital stay.^{1,4}
3. The specimen is ideally passed into a bedpan but must not be contaminated with urine. Alternatively, a large plastic bag or "saran wrap" may be placed over the toilet seat opening and the specimen passed into the bag. A thoroughly cleaned and dried milk carton, cut to remove the upper two third of the carton, may also be used. It will be easier to collect the specimen if the water supply to the toilet is shut off and water drained from the bowl by "flushing twice".
4. An appropriate (i.e. bloody, mucoid, watery) area of stool should be selected and sampled with the collection spoons provided in the caps of the containers. Sufficient stool is added to each container to bring the liquid level up to the "Fill to Here" line. This will result in approximately 5 mL of sample. To insure adequate sampling of a formed stool, material should be removed from the sides, ends, and middle of the bolus.
5. Agitate each specimen with the spoon along the sides of the container, tighten the cap and shake firmly to insure that the specimen is adequately mixed. When mixing is completed the specimen should appear homogenous.
6. Return the vials to their container, seal the container, and label appropriately.

TEST PROCEDURE

The Para-Pak SAF Systems lend themselves to a wide variety of procedures in common use. The following discussion is not exhaustive and alternatives may be found in the literature cited. While variations may exist from lab to lab, a thorough examination should include at least three steps:

1. Gross examination: Record the presence of blood, worms, mucus, or proglottids.
2. Direct microscopic examination from the SAF preserved specimen:
 - a. Place a clean glass slide on a sheet of newsprint.
 - b. Add a drop of saline (iodine may be substituted if desired) to the slide.
 - c. Add a representative sample of SAF preserved specimen to the drop of saline and mix thoroughly with the collecting spoon. The news print must be just legible through the slide.
 - d. Place a double width coverslip on the suspension and examine immediately.
3. Permanent slide and concentration procedure using the Para-Pak® Macro-Con® or Con-Trate® Stool Concentration System:
 - a. Thoroughly mix the material in the Para-Pak SAF specimen collection vial until it appears homogeneous. The specimen is now ready for processing with the Para-Pak Macro-Con stool concentration system. Follow the Para-Pak Macro-Con package insert, Specimen Processing, steps 1 through 8. Insert the following procedure between steps 8 and 9 of the Macro-Con Specimen Processing section. (For Con-Trate, refer to product package insert.)
 1. Add physiological saline to bring the level of the filtered specimen to the dotted line on the Para-Pak Macro-Con label.
 2. Place a provided screw cap on the conical centrifuge tube and centrifuge for 10 minutes at 500-1000 xg (1800-2200 rpm for most tabletop centrifuges).
 3. Carefully pour off the supernatant fluid.
 4. Mix the sediment with an applicator stick in the saline which drains back from the walls of the tube. A small drop of saline may be added if necessary.
 5. Prepare a smear for permanent staining as follows (6, 7):
 - a. Add a small drop of Mayer's albumin (supplied with each pack of 10 kits) to a clean glass slide and wipe immediately so that only a thin coating remains on the slide. NOTE: If slides have a reddish tinge after decolorization, too much albumin has remained on the slide.
 - b. Add a small amount of the suspended sediment to the albumin coated slide.
 - c. Spread the mixture over the slide to produce a film which varies in thickness.
 - d. Allow the slide to dry 5-10 minutes at room temperature until "tacky".
 - e. Immerse the slide in 70% ethanol for approximately 30 minutes to coagulate the albumin.
 - f. Proceed with the staining regimen of choice. Wheatley's Trichrome (Meridian Catalog No. 400101) works particularly well with this method. Omit steps 1-3 in the staining section of the Wheatley's Trichrome package insert.
 - b. Use the remaining sediment to follow Specimen Processing steps 9-18 in the Para-Pak Macro-Con package insert. If the Para-Pak Macro-Con is unavailable the following procedure for permanent slides and concentration may be implemented:
 1. Permanent slide preparation:
 - a. Thoroughly mix the material in the SAF tube until it appears homogenous.
 - b. Strain approximately 2-3 mL (amount will vary in direct proportion to the density of the suspension) of the suspended material through one layer of narrow mesh or two layers of wide mesh gauze into a suitable centrifuge tube.
 - c. Add about 10 mL of physiological saline to the tube and mix thoroughly.
 - d. Centrifuge 10 minutes at 500 xg.
 - e. Decant the supernatant fluid. Approximately 0.5 to 1.0 mL of sediment should remain. If necessary, remove sediment or add more strained suspension and repeat steps c and d.
 - f. Mix the sediment and prepare the permanent smear as described above, step 5a-f.
 2. Use the remaining sediment in the concentration procedure and follow step 3b listed above.

NOTE: The zinc sulfate flotation procedure is reported to give poor results with this fixative and is not recommended. The flotation procedure may, however, be applied to the unpreserved specimen provided it is relatively fresh.⁷

QUALITY CONTROL

This test should be performed per applicable local, state, or federal regulations or accrediting agencies.

The Para-Pak SAF vial should contain approximately 15 mL of fluid to insure 1:3 stool to preservative ratio. When SAF fixed film of stock trophozoite or human buffy coat is stained the organisms or cells should appear well fixed and defined.

If the expected control reactions are not observed, repeat the control tests as the first step in determining the root cause of the failure. If control failures are repeated please contact Meridian's Technical Services Department at 1-800-343-3858 (US) or your local distributor.

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REFERENCES

1. Brooke MM. Intestinal and urogenital protozoa, Manual of Clinical Microbiology, ASM, Washington, D.C., 2nd ed., 1974;582-601.
2. Burrows RB. Microscopic diagnosis of the parasites of man, Yale University Press, New Haven, 1965;319-381.
3. Junod L. Technique coprologique nouvelle essentiellement destinée à la concentration des trophozoites d'amibes, Bul Soc Pathol Exot. 1972;65:390-398.
4. Melvin DM and Brooke MM. Laboratory procedures for the diagnosis of intestinal parasites, U.S.D.H.E.W., 79:8282, CDC, Atlanta, GA, 1980;23-65.
5. Scholten T, and Yang J. Evaluation of preserved and unpreserved stools for the detection and identification of intestinal parasites, Am J Clin Pathol. 1974;62:563-567.
6. Scholten T. An improved technique for the recovery of intestinal protozoa, J Parasitol. 1972;58:633-634.
7. Scholten T. (Personal Communication).
8. Simitch T and Petrovich Z. Longevite de la forme vegetative de dysenterie dans divers milieux. Arch. Inst. Pasteur d'Algérie, 1953;31:375-380.
9. Schwartzwalder, J. Clyde, GW. Hunter, WW. Frye. Manual of tropical medicine, WB. Saunders Co., Philadelphia. 1966.
10. Yang J and Scholten T. A fixative for intestinal parasites permitting the use of concentration and permanent staining procedures, Am J Clin Pathol. 1977;67:300-304.
11. Erdman, Dean. Clinical comparison of ethyl acetate and diethyl ether in the formalin-ether sedimentation technique. J Clin Microbiol. 1981;14:483-485.
12. Young Kirk H et. al. Ethyl acetate as a substitute for diethyl ether in the formalin-ether sedimentation technique. J Clin Microbiol. 1979;10:852-853

SN10913

REV. 11/22

Manufactured By

Meridian Bioscience, Inc.

3471 River Hills Drive
Cincinnati, OHIO - 45244 USA
www.meridianbioscience.com

Contacts:

Main Telephone (+1) 513.271.3700
Customer Service/Orders 800.543.1980
Technical Support Center 800.343.3858
Information Fax: 513.272.5432
Ordering Fax: 513.271.0124
E-mail: info@meridianbioscience.com

INTERNATIONAL SYMBOL USAGE

You may see one or more of these symbols on the labeling/packaging of this product:

Key guide to symbols (Guida ai simboli, Guide des symboles, Guia de símbolos, Zeichenerklärung)

	Use By / Utilizzare entro / Utiliser jusque / Fecha de caducidad / Verwendbar bis	CONTROL +	Positive control / Controllo positivo / Contrôle positif / Control positivo / Positive Kontrolle
LOT	Batch Code / Codice del lotto / Code du lot / Código de lote / chargebezeichnung	CONTROL -	Negative control / Controllo negativo / Contrôle négatif / Control negativo / Negative Kontrolle
IVD	In vitro diagnostic medical device / Dispositivo medico-diagnostico in vitro / Dispositif médical de diagnostic in vitro / Dispositivo médico para diagnóstico in vitro / In-Vitro-Diagnostikum	EC REP	Authorized representative in the European Community / Rappresentante Autorizzato nella Comunità Europea / Mandataire dans la Communauté européenne / Representante autorizado en la Comunidad Europea / Bevollmächtigter in der Europäischen Gemeinschaft
CE	Meridian products carrying the European Conformity (CE) mark fulfill the requirements of Directive 98/79/EC or the Regulation 2017/746 on in-vitro diagnostic medical devices / i prodotti Meridian recanti il marchio di Conformità Europea (CE) soddisfano i requisiti della Direttiva 98/79/CE o del Regolamento 2017/746 sui dispositivi medico-diagnostici in vitro / Les produits Meridian portant la marque de Conformité européenne (CE) sont conformes aux exigences de la Directive 98/79/CE ou du Règlement 2017/746 relatifs aux dispositifs médicaux de diagnostic in vitro / Los productos de Meridian que llevan la marca de conformidad europea (CE) cumplen los requisitos de la Directiva 98/79/CE o del Reglamento 2017/746 sobre los productos sanitarios para diagnóstico in vitro / Produkte von Meridian mit der CE-Kennzeichnung erfüllen die Anforderungen der EU-Richtlinie 98/79/EG bzw. der Verordnung (EU) 2017/746 über In-vitro-Diagnostika	SMP PREP DIL SPE	Sample Preparation Apparatus containing Sample Diluent / Dispositivo per la preparazione del campione contenente il diluente del campione / Système pour la préparation de l'échantillon, diluant inclus / Aparato para Preparación de Muestra con Diluyente de Muestra / System zur Probenvorbereitung, in dem sich Probenverdünnpuffer befindet
REF	Catalogue number / Numero di catalogo / Référence du catalogue / Numero de catálogo / Bestellnummer		CAUTION: Risk of Danger / ATTENZIONE: Pericolo / AVERTISSEMENT: Risques de danger / Precaución: Peligroso / WARNUNG: Risikogefahr
	Consult Instructions for Use / Consultare le istruzioni per l'uso / Consulter les instructions d'utilisation / Consulte las instrucciones de uso / Gebrauchsanweisung beachten	BUF RXN	Reaction Buffer / Tampone di reazione / Solution de réaction tamponnée / Tampón de Reacción / Reaktionspuffer
	Manufacturer / Fabbricanto / Fabricant / Fabricante / Hersteller		For IVD Performance Evaluation Only / Soltanto per valutazione delle prestazioni / Réactifs IVD réservés à l'évaluation des performances / Sólo para evaluación del funcionamiento / Nur zur IVD Leistungsbewertung
	Contains sufficient for <n> tests / Contenuto sufficiente per "n" saggi / Contenu suffisant pour "n" test / Contenido suficiente para <n> ensayos / Inhalt ausreichend für <n> Prüfungen	SOLN STOP	Stopping Solution / Soluzione di Stop / Solution d'arrêt / Solución de parada / Stopplösung
	Temperature limitaion / Limiti di temperatura / Limites de température / Limite de tempperatura / Temperaturbegrenzung	CONJ ENZ	Enzyme Conjugate / Coniugato enzimatico / Conjugué enzymatique / Conjunto enzimático / enzymkonjugat
SN	Serial number / Numero di serie / Numéro de série / Número de serie / Seriennummer	CONTROL	Assay Control / Controllo del test / Test de contrôle / Control de Ensayo / Kontrolltest
TEST	Test Device / Dispositivo test / Dispositif de test / Dispositivo de Prueba / testgarät	REAG	Reagent / Reagente / Réactifs / Reactivos / Reagenzien
	Date of manufacture / Data di fabbricazione / Date de fabrication / Fecha de fabricación / Herstellungsdatum	BUF WASH	Wash Buffer / Soluzione di lavaggio / Solution de lavage / Tampón de lavado / Waschpuffer
BUF	Buffer / Soluzione tampone / Solution tamponnée / Tampón / Puffer		Warning / Avvertenze / Mise En Garde / Advertencia / Warnhinweise
CONJ	Conjugate / Coniugato / Conjugué / Conjunto / Konjugat	DIL SPE	Specimen Diluent (or Sample Diluent) / Diluente del Campione / Diluant échantillons / Diluyente de muestra / Probenverdünnungspuffer
SUBS	Substrate / Substrato / Substrat / Substrato / Substrat	BUF WASH 20X	Wash Buffer Concentration 20X / Soluzione dil lavaggio 20X / Solution de lavage concentrée 20X / Solución tampón de lavado 20X / 20fach konzentriertes Waschkonzentrat
Rx Only	Prescription Use Only / Per l'uso su prescrizione medica / Uniquement sur prescription / Solo Para Uso Por Receta / verschreibungspflichtig	DET REAG	Detection Reagent / Reagente Diretto / Réactif de Detection / Reactivo de Detección / Nachweis Reagenz
	Do not use if package is damaged / Non utilizzare se la confezione è danneggiata / ne pas utiliser si le paquet est endommagé / No use si el paquete esta dañado / Nicht verwenden, wenn die Verpackung beschädigt ist	TUBE	Empty Tube / Prova vuota / Tube vide / Tubo vacío / Leeres Gefäß
	Single Use Only / Prodotto Monouso / A usage unique / Para Un Solo Uso / nur für die einmalige Anwendung		

For technical assistance, call Technical Support Services at 800-343-3858 between the hours of 8AM and 6PM, USA Eastern Standard Time. To place an order, call Customer Service Department at 800-543-1980.