Para-Pak® C&S Para-Pak® Enteric Plus

REF330284, 330384, 331184,
900812, 990620**IVD****Rx Only****INTENDED USE**

The Para-Pak C&S (Culture and Sensitivity) and Para-Pak Enteric Plus provide a standardized procedure for the routine collection, transportation, preservation, and culture of stool specimens for bacterial enteric pathogens. It is designed for easy collection of specimens by individuals not trained in microbiological techniques and affords an excellent means of minimizing the adverse effects of delay in specimen transportation.

SUMMARY AND EXPLANATION OF THE TEST

Diagnosis of intestinal disease of bacterial origin is confirmed by isolation and identification of the disease agent (such as such pathogens as *Salmonella*, *Shigella*, *Campylobacter*, *Cholera Vibrio* and *Yersinia*) from a fresh specimen in the clinical microbiology laboratory. Best results are assured when fresh stool specimens are processed immediately in the laboratory. When delayed transportation or conflicting priorities prevent immediate processing of fresh specimens, the bacterial population must be maintained. Timely collection and transportation of "fresh" stool specimens to the laboratory cannot always be insured. Workload conditions and priorities in clinical laboratories frequently do not permit immediate culture of "fresh" specimens. Procedures such as incubation, refrigeration, or allowing the specimen to stand at room temperature will not guarantee the recovery of all enteric pathogens. Certain pathogens (esp. *Shigella*) show a marked decrease in colony forming units (CFU) within minutes of stool passage.^{2-4, 6, 10}

A method of preserving typhoid stools for delayed culture was described by Teague and Clurman in 1916.¹¹ In 1939, Sachs reported on a modification of the Teague and Clurman solution for increased isolation of *Shigella*, *Salmonella* and cholera vibrios. The efficiency of the buffered glycerol saline (BGS) solution has been confirmed by several authors.^{3, 5-10}

In 1955, Hanja reported on a specimen preservative which doubled isolations of *shigellae* and *salmonella* compared to BGS.³ Recent concern regarding *Campylobacter jejuni* has prompted research demonstrating rapid death of this organism in BGS. An average three log drop in CFU's/mL at 25 C in seeded specimens initially containing 10⁸ – 10⁹ CFU/mL was observed.¹² Proper use of the Para-Pak C&S and Para-Pak Enteric Plus Systems assures the microbiologist of preservation of all known enteric pathogens for 96 hours,¹² and limits overgrowth of these agents by normal flora.

Conditions known to adversely affect survival and recovery of enteric bacterial pathogens include: overgrowth by commensal flora, excessive drying, extremes in temperature, pH shifts, and for *Campylobacter*, lack of a partially reduced environment. These conditions occur when stool specimens are incubated, refrigerated, stored for prolonged periods at ambient temperature, or allowed to dehydrate.^{3, 5, 10} Several transport/preservative systems have been described that improve – with varying degrees of success – recovery of enteric pathogens from fecal specimens.^{1, 2, 4, 7, 10}

Para-Pak C&S and Para-Pak Enteric Plus are designed to facilitate the collection and transport of fecal specimens. The holding medium is formulated to assure optimum recovery of *Salmonella*, *Shigella*, *Campylobacter*, *Vibrio*, and *Yersinia* for 96 hours.

BIOLOGICAL PRINCIPLES

Para-Pak C&S and Para-Pak Enteric Plus are modifications of Cary-Blair, which utilizes an isotonic non-nutritive, buffered solution to insure survival of bacterial pathogens and prevent overgrowth by commensal organisms. A pH indicator is incorporated to indicate acidic conditions in the tube which are not optimal for recovery of the organisms in question.

REAGENTS/MATERIALS PROVIDED

The maximum number of tests obtained from this test kit is listed on the outer box.

Each collection system consists of one vial containing 15 mL of non-nutritive stool transport solution for enteric pathogens. A collection spoon is built into each vial and easy-to-follow collection instructions are provided for patients and nursing staff.

Catalog# 33028410% Buffered Neutral Formalin (item# 9004)
LV-PVA (item# 9001)
C&S (item# 9006)**Catalog# 330384**SAF (item# 9002)
Clean Vials (item# 9003)
C&S (item# 9006)
Mayer's Albumin (item# 9007)**Catalog# 331184**10% Buffered Neutral Formalin (item# 9004)
Zn-PVA (item# 9012)
C&S: (item# 9006)**Catalog# 900812**

Enteric Plus (item# 9008)

Catalog# 990620

C&S (item# 9006)

Active Ingredients:

1. Agar
2. Phosphate Buffer
3. Sodium Chloride
4. Thioglycolic Acid Sodium Salt
5. Calcium Chloride
6. Phenol Red
7. Water

PRECAUTIONS

1. All reagents are for in vitro diagnostic use only.
2. When handling potentially infectious material always practice good hand washing technique and disinfect or sterilize vials prior to disposal.
3. Any vials in which the solutions become cloudy or yellow during storage should be discarded.
4. Occasionally a specimen will be encountered which exceeds the buffering capacity of the solution. This will be indicated by an acid (yellow) shift by the pH indicator. In this case conditions for the survival of enteric pathogens are not optimal. A second specimen should be collected to insure optimal conditions.
5. Shipment of the Para-Pak C&S vial requires a suitable double-walled container to comply with postal regulations and the CDC publication entitled "Packaging and Shipping Biological Materials", U.S. Department of Health and Human Services, Office of Biosafety, Atlanta, Georgia – July, 1981.
6. Collection, transport, and preservation of fecal specimens for culture examination are only an integral part of the overall scheme for collection, transport, propagation, identification, and antimicrobial susceptibility testing of enteric pathogens. Additional tests and procedures may be found in appropriate references.³⁻⁵
7. Any serious incident that has occurred in relation to the device should be reported to Meridian Bioscience, Inc., 3471 River Hills Drive, Cincinnati, Ohio 45244 USA or Technical Support Center 800.343-3858.
8. IMPORTANT: See SDS for additional safety and hazard information.

WARNING

This product must not be used if:

1. The expiration date on the label has passed.
2. Proper storage conditions have not been observed.
3. Quality control performance does not meet specified standards.
4. The contents of the vial are cloudy or yellow.

HAZARD and PRECAUTIONARY STATEMENTS

 <p>Para-Pak LV-PVA</p>	<p>Signal Word Danger</p> <p>Hazard statements H301 - Toxic if swallowed H311 - Toxic in contact with skin H314 - Causes severe skin burns and eye damage H331 - Toxic if inhaled H336 - May cause drowsiness or dizziness H341 - Suspected of causing genetic defects H361 - Suspected of damaging fertility or the unborn child H373 - May cause damage to organs through prolonged or repeated exposure H411 - Toxic to aquatic life with long lasting effects H225 - Highly flammable liquid and vapor Contains Isopropyl alcohol, Acetic acid, Mercuric chloride, Hydrochloric acid</p> <p>Precautionary Statements - EU (§28, 1272/2008) P301 + P310 - IF SWALLOWED: Immediately call a POISON CENTER or doctor/ physician P321 - Specific treatment (see supplemental first aid instructions with this material) P280 - Wear protective gloves/ protective clothing P322 - Specific measures (see ? on this label) P321 - See SDS Section 4 or Section 11 for specific medical treatment information P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P260 - Do not breathe dust/fume/gas/mist/vapors/spray P280 - Wear protective gloves/ protective clothing/ eye protection/ face protection P303 + P361 + P353 - IF ON SKIN (or hair): Remove/ Take off immediately all contaminated clothing. Rinse skin with water/shower P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing P310 - Immediately call a POISON CENTER or doctor/ physician P280 - Wear eye protection/ face protection P281 - Use personal protective equipment as required P201 - Obtain special instructions before use P202 - Do not handle until all safety precautions have been read and understood P308 + P313 - IF exposed or concerned: Get medical advice/ attention P314 - Get medical advice/ attention if you feel unwell P501 - Dispose of contents/ container to an approved waste disposal plant. P210 - Keep away from heat/sparks/open flames/hot surfaces. - No smoking P370 + P378 - In case of fire: Use dry sand, dry chemical or alcohol-resistant foam for extinction</p>
--	---

 <p>Para-Pak SAF</p>	<p>Signal Word Danger</p> <p>Hazard Statements H301 - Toxic if swallowed H311 - Toxic in contact with skin H318 - Causes serious eye damage H341 - Suspected of causing genetic defects H350 - May cause cancer H315 - Causes skin irritation H330 - Fatal if inhaled Contains Formaldehyde,</p> <p>Precautionary Statements - EU (§28, 1272/2008) P301 + P310 - IF SWALLOWED: Immediately call a POISON CENTER or doctor/ physician P321 - Specific treatment (see supplemental first aid instructions with this material) P321 - See SDS Section 4 or Section 11 for specific medical treatment information P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing P310 - Immediately call a POISON CENTER or doctor/ physician P281 - Use personal protective equipment as required P201 - Obtain special instructions before use P308 + P313 - IF exposed or concerned: Get medical advice/ attention P304 + P340 - IF INHALED: Remove to fresh air and keep at rest in a position comfortable for breathing P310 - Immediately call a POISON CENTER or doctor/ physician P302 + P352 - IF ON SKIN: Wash with plenty of soap and water P310 - Immediately call a POISON CENTER or doctor/ physician P362 - Take off contaminated clothing and wash before reuse P308 + P313 - IF exposed or concerned: Get medical advice/ attention P363 - Wash contaminated clothing before reuse P333 + P313 - If skin irritation or rash occurs: Get medical advice/ attention P330 - Rinse mouth P405 - Store locked up P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P501 - Dispose of contents/ container to an approved waste disposal plant. P280 - Wear protective gloves/ protective clothing/ eye protection/ face protection P202 - Do not handle until all safety precautions have been read and understood P264 - Wash face, hands and any exposed skin thoroughly after handling P270 - Do not eat, drink or smoke when using this product P260 - Do not breathe dust/fume/gas/mist/vapors/spray P271 - Use only outdoors or in a well-ventilated area P284 - Wear respiratory protection P272 - Contaminated work clothing should not be allowed out of the workplace</p>
--	---

 <p>Para-Pak Zn-PVA Fixative</p>	<p>Signal Word Danger</p> <p>Hazard Statements H302 - Harmful if swallowed H411 - Toxic to aquatic life with long lasting effects H314 - Causes severe skin burns and eye damage H330 - Fatal if inhaled H350 - May cause cancer H225 - Highly flammable liquid and vapor Contains Zinc sulfate, Isopropyl alcohol, Methyl alcohol, Acetic acid</p> <p>Precautionary Statements - EU (§28, 1272/2008) P280 - Wear eye protection/ face protection P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing P310 - Immediately call a POISON CENTER or doctor/ physician P210 - Keep away from heat/sparks/open flames/hot surfaces. - No smoking P370 + P378 - In case of fire: Use dry sand, dry chemical or alcohol-resistant foam for extinction P304 + P340 - IF INHALED: Remove to fresh air and keep at rest in a position comfortable for breathing P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P260 - Do not breathe dust/fume/gas/mist/vapors/spray P303 + P361 + P353 - IF ON SKIN (or hair): Remove/ Take off immediately all contaminated clothing. Rinse skin with water/shower P201 - Obtain special instructions before use P281 - Use personal protective equipment as required P308 + P313 - IF exposed or concerned: Get medical advice/ attention</p>
---	---

 <p>Para-Pak 10% Formalin</p>	<p>Signal Word Danger</p> <p>Hazard Statements H301 - Toxic if swallowed H311 - Toxic in contact with skin H317 - May cause an allergic skin reaction H341 - Suspected of causing genetic defects H350 - May cause cancer H402 - Harmful to aquatic life H330 - Fatal if inhaled H315 - Causes skin irritation H318 - Causes serious eye damage H370 - Causes damage to organs</p> <p>Precautionary Statements - EU (§28, 1272/2008) P301 + P310 - IF SWALLOWED: Immediately call a POISON CENTER or doctor/ physician P321 - Specific treatment (see supplemental first aid instructions with this material) P280 - Wear protective gloves/ protective clothing P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P280 - Wear eye protection/ face protection P321 - See SDS Section 4 or Section 11 for specific medical treatment information P201 - Obtain special instructions before use P281 - Use personal protective equipment as required P308 + P313 - IF exposed or concerned: Get medical advice/ attention P202 - Do not handle until all safety precautions have been read and understood P264 - Wash face, hands and any exposed skin thoroughly after handling P270 - Do not eat, drink or smoke when using this product P260 - Do not breathe dust/fume/gas/mist/vapors/spray P271 - Use only outdoors or in a well-ventilated area P284 - Wear respiratory protection P272 - Contaminated work clothing should not be allowed out of the workplace P307 + P311 - IF exposed: Call a POISON CENTER or doctor/ physician P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing P310 - Immediately call a POISON CENTER or doctor/ physician P302 + P352 - IF ON SKIN: Wash with plenty of soap and water P312 - Call a POISON CENTER or doctor/ physician if you feel unwell P361 - Remove/ Take off immediately all contaminated clothing P332 + P313 - If skin irritation occurs: Get medical advice/ attention P363 - Wash contaminated clothing before reuse P304 + P340 - IF INHALED: Remove to fresh air and keep at rest in a position comfortable for breathing P310 - Immediately call a POISON CENTER or doctor/ physician P405 - Store locked up P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P501 - Dispose of contents/ container to an approved waste disposal plant.</p>
--	---

SHELF LIFE AND STORAGE

Shelf life of Para-Pak C&S and Para-Pak Enteric Plus are indicated on the outer package label. Store at 2-30 C. Excessive heat and cold should be avoided.

SPECIMEN COLLECTION AND PREPARATION

- The patient should be cautioned against the use of antacids, barium, bismuth, anti-diarrheal medication or oily laxatives prior to collection of the specimen.
- To assure the recovery of enteric pathogens numerous authors have cited the value of multiple specimens. Three consecutive specimens obtained during the acute stage (first three days) of diarrheal disease are suggested.^{3,4, 6, 8, 9}
- A bed pan is an ideal initial collection container provided it has been thoroughly cleaned and the patient is cautioned against contaminating the specimen with urine. A clean, wide-mouthed container or a plastic bag placed over the toilet seat are also acceptable.
- An appropriate (ie, bloody, slimy, watery) area of stool should be selected and sampled with the collection spoon provided in the cap of the container. Sufficient stool is added to each container to bring the liquid level up to the "fill to here" line. This will result in introducing approximately 1 gram of sample into the preservative solution.
- Mash and mix the stool against the side of the vial with the spoon. Tighten the cap and shake the vial until the mixture appears homogeneous.
- Label the vial with pertinent patient information. Include the time and date of collection. Transport the specimen to the laboratory. Keep the specimen at room temperature unless instructed to do otherwise by the in vitro diagnostic device with which the sample will be used.

TEST PROCEDURE

If on arrival in the laboratory the buffering capacity of the solution has been exceeded (indicated by a yellow colored solution) another specimen should be obtained.

Because of the potential for encountering a variety of bacterial, viral and parasitic agents in stool specimens, it is suggested that disposable plastic gloves be worn during subculture and that the subculture is performed as follows:

- Don disposable plastic gloves.
- Prepare a shallow tray by placing paper toweling in the bottom and applying sufficient disinfectant to just wet the toweling.
- Swirl the vial gently to resuspend the stool.
- Tap the vial lightly on the counter, octagonal end down to remove any liquid which may be adhering to the cap.
- Stand the vial upright on the wet toweling.
- Remove the cap and lay it on paper toweling saturated with disinfectant.
- Using an inoculation loop, swab or pipette, remove a portion of the holding media/stool mixture and inoculate enrichment, selective and/or differential media as indicated by laboratory procedure.
- Reseal the vial tightly. The specimen vial, and any subsequently contaminated material should be sterilized before disposal, as dictated by good laboratory safety practice.

A specimen submitted in Para-Pak C&S or Para-Pak Enteric Plus may be cultured routinely to any of a variety of selective and differential media or used in diagnostic procedures. For cultures, the user is referred to one or more of the general references under REFERENCES for a complete discussion of the various regimens available.

QUALITY CONTROL

This test should be performed per applicable local, state, or federal regulations or accrediting agencies.

- Visual inspection: vials should contain approximately 15 mL of red fluid.
 - The holding medium must be red in color and visually free of particulate matter.
 - The medium must be capable of maintaining sufficient bacterial population of a normal stool specimen for up to 72 hours*.
- *Extremely fastidious organisms such as *C. jejuni* may exhibit minimal recovery after 48 hours.

If the expected control reactions are not observed, repeat the control tests as the first step in determining the root cause of the failure. If control failures are repeated please contact Meridian's Technical Services Department at 1-800-343-3858 (US) or your local distributor.

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REFERENCES

1. Amies CR. Evaluation of a simple preservative for bacteriological tests on urine. *Can Med Assoc J* 1973;108:469-471.
2. Amies CR and Corpas A. A preservative for urine specimens in transit to the bacteriological laboratory. *J Med Microbiol* 1971;4:362-365.
3. Edwards PR and Ewing WJ. Identification of enterobacteriaceae. 3rd ed. Minneapolis: Burgess Publishing Company; 1972. p. 28, 337-338.
4. Eisengerg HD, Schoenknecht FD and Von Gravenitz A. Cumitech 9-collection and processing of bacterial specimens. Washington (DC): ASM; 1979.
5. Ewing WJ, McWhorter AC and Montague TS. Transport media in the detection of *Salmonella typhi* in carriers. *Pub Health Lab*; 1966. 24:63-65.
6. Ewing WH. Transport methods for Enterobacteriaceae and allied bacterial. *Pub Health Lab*; 1971.29:8-23.
7. Finegold SM, Martin WJ and Scott EG. Diagnostic microbiology. St. Louis: CV Mosby Co; 1978. p. 148-152.
8. Kelly MT, Brenner DJ and Farmer III JJ. Enterobacteriaceae in manual of clinical microbiology. 4th ed. Washington (DC): ASM; 1985. p. 263-264.
9. Sachs A. Difficulties associated with the bacteriological diagnosis of bacillary dysentery. *J. Royal Army Medical Corps*; 1939. 73:234-235.
10. Sonnenwirth AC. Collections and culture of specimens and guides for bacterial identification, in gradwohl's clinical laboratory methods and diagnosis. 7th ed. St. Louis: CH Mosby Co; 1970. p. 1149-1152.
11. Teague O. and Clurman AV. A method of preserving typhoid stools for delayed examination and a comparative study of the efficiency of each brilliant-green agar eosin methylene blue agar and endo agar for isolation of typhoid bacilli from stools. *J Infect Dis* 1916;18:653-671.
12. Data on file. Meridian Bioscience, Inc. Cincinnati, OH 45244.

SN10852

REV. 11/22

 <p>Manufactured By</p>	<p>Meridian Bioscience, Inc. 3471 River Hills Drive Cincinnati, OHIO - 45244 USA www.meridianbioscience.com</p> <p><u>Contacts:</u> Main Telephone (+1) 513.271.3700 Customer Service/Orders 800.543.1980 Technical Support Center 800.343.3858 Information Fax: 513.272.5432 Ordering Fax: 513.271.0124 E-mail: info@meridianbioscience.com</p>
--	--

INTERNATIONAL SYMBOL USAGE

You may see one or more of these symbols on the labeling/packaging of this product:

Key guide to symbols (Guida ai simboli, Guide des symboles, Guía de símbolos, Zeichenerklärung)

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