Meridian Viral Transport

REF 505021

IVD

Rx Only

INTENDED USE

Meridian Viral Transport is a system which contains all materials necessary to collect and transport clinical specimens for viral isolation.

SUMMARY AND EXPLANATION OF THE TEST

Classically, the method of choice for the demonstration of the presence of virus in patient specimens involves tissue culture isolation. The efficacy of this method depends upon the collection of a specimen containing infectious virus and the maintenance of the infectivity during transport to the testing laboratory.

Many viruses (such as respiratory syncytial virus⁵) are labile and specimens for isolation require a transport media to stabilize their infectivity.¹⁻⁶

BIOLOGICAL PRINCIPLES

Meridian Viral Transport contains a sterile swab for collection of a clinical specimen and a tube containing a stabilizing medium which has been proven effective in sustaining viral infectivity.¹

REAGENTS/MATERIALS PROVIDED

The maximum number of tests obtained from this test kit is listed on the outer box.

Meridian Viral Transport contains all materials needed for collection of routine swab-type specimens.

Contents

1. Sterile swab – Flocked nylon head on a plastic shaft

2. Transport Tube – containing 1.5 mL of medium

Medium formulation per liter:

Tryptose phosphate broth	29.5 g/L
Gelatin	5.0 g/L
Gentamycin sulfate	50.0 mg/L

MATERIALS NOT PROVIDED

Certain types of specimens (eg, lesion scrapings, vesicular fluid) may require the use of additional materials (scalpel, hypodermic needle). The materials that will be required depend on the type of the specimen and the methodology of the physician.

PRECAUTIONS

1. All reagents are for in vitro diagnostic use only.
2. Meridian Viral Transport tubes containing patient specimens should be considered potentially infectious and handled accordingly. All clinical specimens should be autoclaved before disposal.

HAZARD and PRECAUTIONARY STATEMENTS

There are no known hazards associated with this product.

SHELF LIFE AND STORAGE

Store at 2-25 C before use. Inspect each tube visually before use. DO NOT USE tubes showing evidence of contamination (cloudiness or precipitate). Do not use beyond the labeled expiration date on the outer box label.

TEST PROCEDURE

Specimen Collection

1. Peel open the package and remove the sterile swab. Collect a patient specimen with the swab employing standard clinical procedures.¹⁻⁶
2. Remove the cap from the tube. Insert the inoculated swab and break off the swab shaft at score mark by bending shaft against the tube rim.
3. Replace the cap on the tube and seat it securely.
4. Label tube with patient identification and forward it to the laboratory for testing.

Specimen Transport

The mode of transport of patient specimens to the testing laboratory depends upon the virus being sought. The following is intended as a general guideline applicable to Herpes Simplex, as well as a number of other clinically relevant viruses. (See also section on LIMITATIONS)

1. Specimens should be placed promptly in Meridian Viral Transport for best results.
2. Specimens should be held at refrigerator temperature (2-8 C) until they reach the testing laboratory.
3. If specimens are to be delayed longer than 72 hours, they should be frozen and shipped at ≤ -70 C. (See section on LIMITATIONS for further information regarding storage.)
Note: Some virus infectivity will be lost upon freezing. Avoid repeated freezing and thawing or storage in self-defrosting freezers.
4. Leftover specimens can be stored frozen at ≤ -70 C for future reference.

QUALITY CONTROL

This test should be performed per applicable local, state, or federal regulations or accrediting agencies.

The preservation capabilities of each lot of Meridian Viral Transport are confirmed before release in tests for Herpes Simplex Virus (HSV) infectivity. For most applications, no additional controls need be performed by the user.

LIMITATIONS OF THE PROCEDURE

1. The transport and storage of many viruses requires special handling. The user should consult the standard references listed in the References for additional information regarding proper transportation conditions.
2. Viral transport media may be employed for collecting specimens for viral detection assays. Viral detection assays may have more stringent collection, storage and handling requirements than defined in this insert. In such cases, follow the more stringent criteria of the immunoassay. This product was cleared for use with TRU FLU and TRU RSV rapid immunoassays. Consult the package insert for those products for optimum sample use, storage and transport conditions.

EXPECTED VALUES AND SPECIFIC PERFORMANCE CHARACTERISTICS

The tryptose phosphate broth contained in Meridian Viral Transport tubes is a standard broth-based medium for the transport of clinical specimens intended for virus isolation procedures. (See Reference 1.)

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REFERENCES

- Johnson FB. Transport of viral specimens. Clin Microbiol Rev 1990; 3:120-131.
- Smith TF. Specimen requirements: Selection, collection, transport and processing. In: Specter S, Hodinka RL, Young SA, Eds. Clinical virology manual, ed 3. Washington DC, ASM Press; 2000; pp11-26.
- Forman MS, Valsamakis A. Specimen collection, transport and processing: virology. In: Murray PR et al. Eds. Manual of Clinical microbiology, ed 9. Washington DC: ASM Press, 2007; pp 1284-1296.
- Atmar RL. Influenza viruses. In: Murray PR et al. Eds. Manual of Clinical microbiology, ed 9. Washington DC: ASM Press, 2007; pp 1340-1351.
- Tang Y-M, Crowe JE Jr. Respiratory syncytial virus and human metapneumovirus. In: Murray PR et al. Eds. Manual of Clinical microbiology, ed 9. Washington DC: ASM Press, 2007; pp 1361-1377.
- Jerome KR, Morrow RA. Herpes simplex viruses and herpes B virus. In: Murray PR et al. Eds. Manual of Clinical microbiology, ed 9. Washington DC: ASM Press, 2007; pp 1523-1531.

SN10867

REV. 04/17

<p>Manufactured By</p>	<p>Meridian Bioscience, Inc. Corporate Office 3471 River Hills Drive Cincinnati, Ohio 45244 USA Telephone: 513.271.3700 Orders/Customer Service: 800.543.1980 Technical Support Center: 800.343.3858 Information Fax: 513.272.5432 Ordering Fax: 513.271.0124</p>
	<p>Meridian Bioscience Europe S. r. l. Via dell'Industria, 7 20020 Villa Cortese, Milano ITALY Tel: +39 0331 43 36 36 Fax: +39 0331 43 36 16 Email: info@meridianbioscience.eu WEB: www.meridianbioscience.eu</p>

Meridian Bioscience Europe s.a./n.v.
2 Avenue du Japon - 1420 Braine l'Alleud
BELGIUM
Tel: +32 (0) 67 89 59 59
Fax: +32 (0) 67 89 59 58
Email: info.bn@meridianbioscience.eu

Meridian Bioscience Europe France
34 rue de Ponthieu - 75008 Paris
FRANCE
Tel: +33 (0) 1 42 56 04 40
Fax: +33 (0) 9 70 06 62 10
Email: info.fr@meridianbioscience.eu

Meridian Bioscience Europe b.v.
Postbus 301 - 5460 AH Veghel
NETHERLANDS
Tel: +31 (0) 411 62 11 66
Fax: +31 (0) 411 62 48 41
Email: info.bn@meridianbioscience.eu

INTERNATIONAL SYMBOL USAGE

You may see one or more of these symbols on the labeling/packaging of this product:

Key guide to symbols (Guida ai simboli, Guide des symboles, Guía de símbolos, Zeichenerklärung)

	Use By / Utilizzare entro / Utiliser jusque / Fecha de caducidad / Verwendbar bis	CONTROL +	Positive control / Controllo positivo / Contrôle positif / Control positivo / Positive Kontrolle
LOT	Batch Code / Codice del lotto / Code du lot / Código de lote / Chargenbezeichnung	CONTROL -	Negative control / Controllo negativo / Contrôle négatif / Control negativo / Negative Kontrolle
IVD	In vitro diagnostic medical device / Dispositivo medico-diagnostico in vitro / Dispositif médical de diagnostic in vitro / Dispositivo medico para diagnóstico in vitro / In-Vitro-Diagnostikum	EC REP	Authority Representative in the European Community / Rappresentante Autorizzato nella Comunità Europea / Mandataire dans la Communauté européenne / Representante autorizado en la Comunidad Europea / Bevollmächtigter in der Europäischen Gemeinschaft
CE	This product fulfills the requirements of Directive 98/79/EC on in vitro diagnostic medical devices / Questo prodotto soddisfa i requisiti della Direttiva 98/79/CE sui dispositivi medico-diagnostici in vitro / Ce produit répond aux exigences de la Directive 98/79 CE relative aux dispositifs médicaux de diagnostic in vitro / Este producto cumple con las exigencias de la Directiva 98/79/CE sobre los productos sanitarios para diagnóstico in vitro / Dieses Produkt entspricht den Anforderungen der Richtlinie über In Vitro Diagnostica 98/79/EG	SMP PREP DIL SPE	Sample Preparation Apparatus containing Sample Diluent / Dispositivo per la preparazione del campione contenente il diluente del campione / Système pour la préparation de l'échantillon diluant inclus / Aparato para Preparación de Muestra con Diluyente de Muestra / System zur Probenzubereitung, in dem sich Probenverdünnungspuffer befindet
REF	Catalogue number / Numero di catalogo / Référence du catalogue / Número de catálogo / Bestellnummer		Do not freeze / Non congelare / Ne pas congeler / No congelar / Nicht Einfrieren
	Consult Instructions for Use / Consultare le istruzioni per l'uso / Consulter les instructions d'utilisation / Consultar las instrucciones de uso / Gebrauchsanweisung beachten	BUF RXN	Reaction Buffer / Tampone di reazione / Solution de réaction tamponnée / Tampón de Reacción / Reaktionspuffer
	Manufacturer / Fabricante / Fabricant / Fabricante / Hersteller		For IVD Performance Evaluation Only / Soltanto per valutazione delle prestazioni / Reagent IVD reserves à l'évaluation des performances / Sólo para evaluación del funcionamiento / Nur zur IVD Leistungsbewertung
	Contains sufficient for <n> tests / Contenuto sufficiente per <n> saggi / Contenu suffisant pour <n> tests / Contenido suficiente para <n> ensayos / Inhalt ausreichend für <n> Prüfungen	SOLN STOP	Stopping Solution / Soluzione di Stop / Solution d'arrêt / Solución de parada / Stopplösung
	Temperature limitation / Limiti di temperatura / Limites de température / Limite de temperatura / Temperaturbegrenzung	CONJ ENZ	Enzyme Conjugate / Coniugato enzimatico / Conjugué enzymatique / Conjugado enzimático / Enzymkonjugat
SN	Serial number / Numero di serie / Numéro de série / Número de serie / Seriennummer	CONTROL	Assay Control / Controllo del test / Test de contrôle / Control de Ensayo / Kontrolltest
TEST	Test Device / Dispositivo test / Dispositif de test / Dispositivo de Prueba / Testgerät	REAG	Reagent / Reagents / Réactifs / Reactivos / Reagenzien
	Date of manufacture / Data di fabbricazione / Date de fabrication / Fecha de fabricación / Herstellungsdatum	BUF WASH	Wash Buffer / Soluzione di lavaggio / Solution de lavage / Tampón de lavado / Waschpuffer
BUF	Buffer / Soluzione tampone / Solution tamponnée / Tampón / Puffer		Warning / Avvertenze / Mise En Garde / Advertencia / Warnhinweise
CONJ	Conjugate / Coniugato / Conjugué / Conjugado / Konjugat	DIL SPE	Specimen Diluent (or Sample Diluent) / Diluente del Campione / Diluant échantillons / Diluyente de muestra / Probenverdünnungspuffer
SUBS	Substrate / Substrato / Substrat / Substrato / Substrat	BUF WASH 20X	Wash Buffer Concentration: 20X / Soluzione di lavaggio 20X / Solution de lavage concentrée 20X / Solución tampón de lavado 20X / 20fach konzentriertes Waschkonzentrat
R. Only	Prescription Use Only / Per l'uso su prescrizione medica / Uniquement sur prescription / Solo Para Uso Por Receta / verschreibungspflichtig	DET REAG	Detection Reagent / Reagente Diretto / Réactif de Détection / Reactivo de Detección / Nachweis Reagens
	Do not use if package is damaged / Non utilizzare se la confezione è danneggiata / Ne pas utiliser si le paquet est endommagé / No use si el paquete está dañado / Nicht verwenden, wenn die Verpackung beschädigt ist		

For technical assistance, call Technical Support Services at 800-343-3858 between the hours of 8AM and 6PM, USA Eastern Standard Time. To place an order, call customer Service Department at 800-543-1980.