
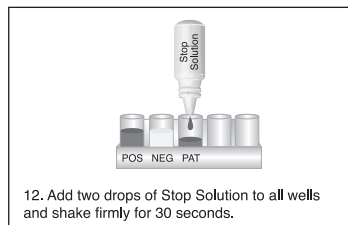
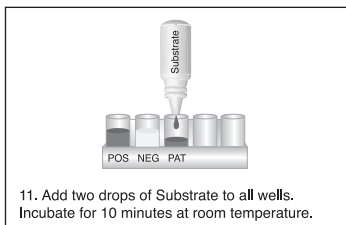
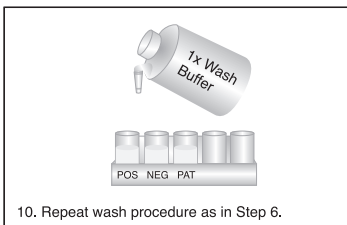
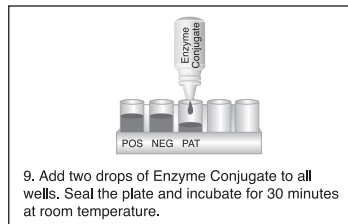
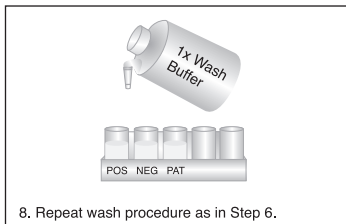
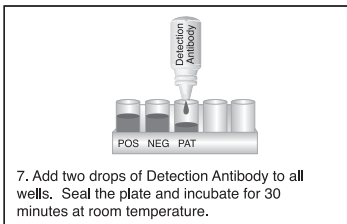
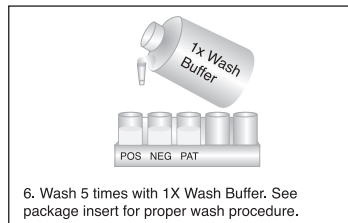
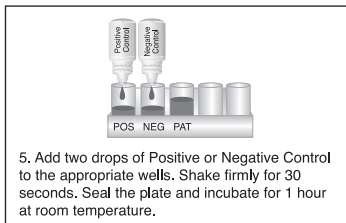
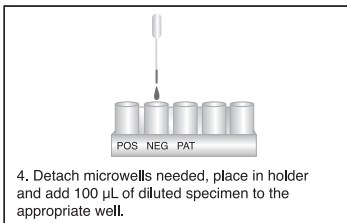
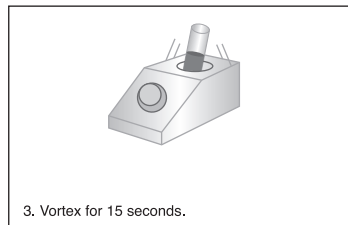
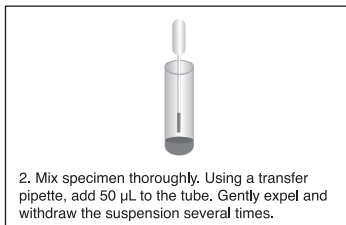


TEST PROCEDURE

How to perform the test

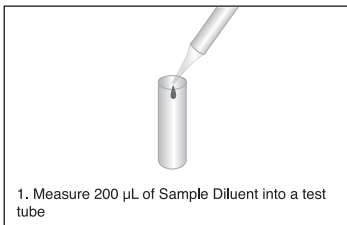

Interpretation of Results: Visual
Negative = colorless
Positive = definite yellow color

Spectrophotometric
Single Wavelength (450 nm)
Negative = OD₄₅₀ < 0.180
Positive = OD₄₅₀ \geq 0.180

Spectrophotometric
Dual Wavelength (450/630 nm)
Negative = OD_{450/630} < 0.150
Positive = OD_{450/630} \geq 0.150

SN11152 REV. 02/18



This illustration is representative of the current Package Insert at the time of publication. Please refer to the most current version of the Package Insert for complete instructions.

USA/CORPORATE OFFICE
3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513.271.3700

Orders / Customer Service: 1.800.543.1980
Technical Support: 1.800.343.3858
Information Fax: 513.272.5432
Ordering Fax: 513.271.0124
www.meridianbioscience.com

MERIDIAN BIOSCIENCE EUROPE

Belgium/Luxembourg Tel: +32 (0) 67 89 59 59
France Tel: +33 (0) 1 42 56 04 40
Germany Tel: +49 3371 6022231
Italy Tel: +39 0331 43 3636
Netherlands Tel: +31 (0) 411 62 11 66
United Kingdom Tel: +44 (0) 20 8453 7970

 **Meridian**
Bioscience, Inc.
Inspired Science. Trusted Solutions.™