# MOCK PIE

For use with the Revogene®

REF 610208

IVD For *in vitro* Diagnostic Use

R<sub>x</sub> Only

**INTENDED USE**  
The MOCK PIE is a re-usable PIE intended for use in conjunction with the Revogene® instrument and Revogene® assays. The MOCK PIE simulates the presence of an assay PIE for thermal and rotational balance when the number of assay PIEs is lower than eight in a run.

Package Contents: Each box contains eight MOCK PIEs in a holder.

#### Principles of the procedure and application

The Revogene rotor requires a balanced load of eight PIEs to run properly. In a situation where less than eight clinical specimens are processed, mock samples containing no target organism, nor negative nor positive controls, may be used.

Meridian Bioscience, Inc. has designed a universal and re-usable MOCK PIE to prevent users from having to prepare mock samples. MOCK PIEs may be used in conjunction with any Revogene assay. The MOCK PIE is designed to behave exactly as an assay PIE in terms of thermal exchange and inertia. MOCK PIEs may also be used when performing a Revogene Self-test (see the Revogene Operator's Manual for more details).

#### INSTRUCTION FOR USE

##### Using the MOCK PIE with assay PIEs

NOTE: Each run must be performed with eight PIEs in the Revogene. When less than eight samples are processed, the empty places can be filled with MOCK PIEs.

1. Before use, ensure MOCK PIE is at room temperature.
2. Proceed to assay set-up according to the corresponding Revogene assay package insert. After all Revogene assay PIEs are inserted into the instrument, fill the empty places with MOCK PIEs. No scan is required when inserting the MOCK PIE into the Revogene. The eight rotor positions must be occupied.
3. Continue by following the corresponding Revogene assay package insert. Refer to the Revogene Operator's Manual for further information regarding Revogene set-up and operation.

##### Viewing and exporting results

The positions occupied by MOCK PIEs will not be displayed on the Results screen and report.

##### After the test run

1. Discard the assay PIE according to the corresponding assay package insert.
2. Remove the MOCK PIEs from the Revogene, clean if necessary and place them back in the holder. Clean according to the cleaning and decontamination procedure listed below.

##### Using the MOCK PIE for a Revogene Self-test

The MOCK PIE can be used for the Revogene Self-test. Refer to the Revogene Operator's Manual for further information regarding the Revogene Self-test.

##### Storage and Handling

Store the MOCK PIE at room temperature. Protect from direct exposure to sunlight.

##### Warnings and Precautions

1. Do not exceed the MOCK PIE's expiration date.
2. Dispose of MOCK PIEs if signs of wear are visible (e.g. chipped edge, delaminated capping).
3. Should any biological liquid be spilled on the MOCK PIE, follow the instructions in the section Cleaning and decontamination.
4. Do not immerse in any liquid including cleaning and decontamination solutions.
5. MOCK PIE must be at room temperature before use.

**Cleaning and decontamination:** The MOCK PIE should be cleaned at least weekly, or daily if it is used frequently. Do not autoclave the MOCK PIE.

##### To clean and decontaminate the MOCK PIE

1. Dampen a lint-free wipe with diluted bleach solution at a concentration of 0,55% sodium hypochlorite and wipe the MOCK PIE. Dispose of lint-free wipe.
2. Wait 10 minutes.
3. Dampen a new lint-free wipe with 70% isopropyl alcohol and wipe the MOCK PIE. Dispose of lint-free wipe.

**Disposal:** Dispose of the MOCK PIE after its expiration date (see MOCK PIE label for expiration date) or if visible signs of wear appear (e.g. chipped edge, delaminated capping). Dispose of waste in accordance with your country, federal, provincial, state and local regulations.

To learn more about the Revogene instrument, please refer to the Revogene Operator's Manual.

**Expiration date:** See MOCK PIE label for expiration date.

##### E-Labeling

Documentation related to this product can be accessed online at [www.meridianbioscience.com/pi](http://www.meridianbioscience.com/pi). Additionally, paper copies are available upon request by contacting your local distributor or via the phone number listed on the kit box.

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REV. 08/20

Manufactured By

**Meridian Bioscience, Inc.**  
Corporate Office  
3471 River Hills Drive  
Cincinnati, Ohio 45244 USA  
Telephone: 513.271.3700  
Orders/Customer Service:  
800.543.1980  
Technical Support Center:  
800.343.3858  
Information Fax: 513.272.5432  
Ordering Fax: 513.271.0124

Authorized Representative

**Meridian Bioscience Europe S. r. l.**  
Via dell'Industria, 7  
**20095** Villa Cortese, Milano  
ITALY  
Tel: +39 0331 43 36 36  
Fax: +39 0331 43 36 16  
Email: [info@meridianbioscience.eu](mailto:info@meridianbioscience.eu)  
WEB: [www.meridianbioscience.com/eu](http://www.meridianbioscience.com/eu)

**Australian Sponsor**  
**Emergo Australia**  
**Level 20, Tower II**  
**Darling Park**  
**201 Sussex Street**  
**Sydney, NSW 2000**  
**Australia**

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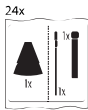
**INTERNATIONAL SYMBOLS USAGE**

You may see one or more of these symbols on the labeling/packaging of this product:

Key guide to symbols (Guida ai simboli, Guide des symboles, Guía de símbolos, Zeichenerklärung)

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For technical assistance, call Technical Support Services at 800-343-3858 between the hours of 8AM and 6PM, USA Eastern Standard Time. To place an order, call Customer Service Department at 800-543-1980.

Revogene and associated logos are registered trademarks of Meridian Bioscience, Inc.  
© 2020-01 Meridian Bioscience, Inc.  
Made in United States

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Revogene e I loghi associate sono marchi commerciali di Meridian Bioscience, Inc.
Revogene et les logos connexes sont des marques déposées de Meridian Bioscience, Inc.
© 2020-001 Meridian Bioscience, Inc.
Revogene así como los logotipos asociados son marcas comerciales de Meridian Bioscience, Inc.
©2020-001 Meridian Bioscience, Inc.
Revogene und die damit verbundenen Logos sind Marken von Meridian Bioscience, Inc.