MOCK PIE

For use with the Revogene®

REF 610208

IVD For *in vitro* Diagnostic Use

R_x Only

INTENDED USE

The MOCK PIE is a re-usable PIE intended for use in conjunction with the 610210, Revogene® instrument and Revogene® assays. The MOCK PIE simulates the presence of an assay PIE for thermal and rotational balance when the number of assay PIEs is lower than eight in a run. The MOCK PIE is intended for use by those trained to operate the automated Revogene test system.

Package Contents: Each box contains eight MOCK PIEs in a holder.

Principles of the procedure and application

The Revogene rotor requires a balanced load of eight PIEs to run properly. In a situation where less than eight clinical specimens are processed, mock samples containing no target organism, nor negative nor positive controls, may be used.

Meridian Bioscience, Inc. has designed a universal and re-usable MOCK PIE to prevent users from having to prepare mock samples. MOCK PIEs may be used in conjunction with any Revogene assay. The MOCK PIE is designed to behave exactly as an assay PIE in terms of thermal exchange and inertia. MOCK PIEs may also be used when performing a Revogene Self-test (see the Revogene Operator's Manual SN134822 for more details).

INSTRUCTION FOR USE

Using the MOCK PIE with assay PIEs

NOTE: Each run must be performed with eight PIEs in the Revogene. When less than eight samples are processed, the empty places can be filled with MOCK PIEs.

1. Before use, ensure MOCK PIE is at room temperature.
2. Proceed to assay set-up according to the corresponding Revogene assay package insert. (Refer to Revogene assay's instructions for use available online at www.meridianbioscience.com/pi). After all Revogene assay PIEs are inserted into the instrument, fill the empty places with MOCK PIEs. No scan is required when inserting the MOCK PIE into the Revogene. The eight rotor positions must be occupied.
3. Continue by following the corresponding Revogene assay package insert. Refer to the Revogene Operator's Manual (SN134822) for further information regarding Revogene set-up and operation.

Viewing and exporting results

The positions occupied by MOCK PIEs will not be displayed on the Results screen and report.

After the test run

1. Discard the assay PIE according to the corresponding assay package insert.
2. Remove the MOCK PIEs from the Revogene, clean if necessary and place them back in the holder. Clean according to the cleaning and decontamination procedure listed below.

Using the MOCK PIE for a Revogene Self-test

The MOCK PIE can be used for the Revogene Self-test. Refer to the Revogene Operator's Manual for further information regarding the Revogene Self-test.

Storage and Handling

Store the MOCK PIE at room temperature. Protect from direct exposure to sunlight.

Warnings and Precautions

1. Do not exceed the MOCK PIE's expiration date.
2. Dispose of MOCK PIEs if signs of wear are visible (e.g. chipped edge, delaminated capping).
3. Should any biological liquid be spilled on the MOCK PIE, follow the instructions in the section Cleaning and decontamination below. Do not autoclave the MOCK PIE.
4. Do not immerse in any liquid including cleaning and decontamination solutions.
5. MOCK PIE must be at room temperature before use.

Any serious incident that has occurred in relation to the device should be reported to Meridian Bioscience, Inc., 3471 River Hills Drive, Cincinnati, Ohio 45244 USA or Technical Support Center 800.343.3858 and competent authority of the EU Member State in which the clinician and/or patient is established.

Cleaning and decontamination: The MOCK PIE should be cleaned at least weekly, or daily if it is used frequently. Do not autoclave the MOCK PIE.

To clean and decontaminate the MOCK PIE

1. Dampen a lint-free wipe with diluted bleach solution at a concentration of 0,55% sodium hypochlorite and wipe the MOCK PIE. Dispose of lint-free wipe.
2. Wait 10 minutes.
3. Dampen a new lint-free wipe with 70% isopropyl alcohol and wipe the MOCK PIE. Dispose of lint-free wipe.

Disposal: Dispose of the MOCK PIE after its expiration date (see MOCK PIE label for expiration date) or if visible signs of wear appear (e.g. chipped edge, delaminated capping). Dispose of waste in accordance with your country, federal, provincial, state and local regulations. To learn more about the Revogene instrument, please refer to the Revogene Operator's Manual.

Expiration date: See MOCK PIE label for expiration date.

E-Labeling

Documentation related to this product can be accessed online at www.meridianbioscience.com/pi. Additionally, paper copies are available upon request by contacting your local distributor or via the phone number listed on the kit box.

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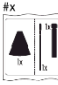
INTERNATIONAL SYMBOLS USAGE

You may see one or more of these symbols on the labeling/packaging of this product:

Key guide to symbols (Guida ai simboli, Guide des symboles, Guia de simbolos, Zeichenerklärung)

	Use-by date / Data di scadenza / Date de péremption / Fecha de caducidad / Verwendbar bis
	Batch code / Codice di lotto / Code de lot / Código de lote / Chargennummer
	In vitro diagnostic medical device / Dispositivo medico diagnostico <i>in vitro</i> / Instrument de test diagnostique <i>in vitro</i> / Producto sanitario para diagnóstico <i>In vitro</i> / <i>In-vitro</i> -Diagnostikum
	CE Mark / Marcatura CE / Symbole CE / Marcado CE / CE-Kennzeichen
	Catalog number / Numero di catalogo / Référence catalogue / Referencia / Bestellnummer
	Consult instructions for use / Consultare le istruzioni per l'uso / Consulter le mode d'emploi / Consultar las instrucciones de uso / Gebrauchsanweisung beachten
	Manufacturer / Produttore / Fabricant / Fabricante / Hersteller
	Contains sufficient for <n> tests / Contiene una quantità sufficiente per <n> test / Contient le matériel suffisant pour <n> tests / Contiene la cantidad suficiente para <n> ensayos / Inhalt ausreichend für <n> Tests
	Temperature limit / Limite di temperatura / Limite de température / Limite de temperatura / Temperaturgrenze
	Authorized representative in the European Community / Rappresentante autorizzato nella Comunità Europea / Représentant agréé dans la Communauté européenne / Representante autorizado en la Comunidad Europea / Bevollmächtigter EU-Repräsentant
	Do not reuse / Non riutilizzare / Ne pas réutiliser / No reutilizar / Nicht wiederverwenden
	Keep dry / Conservare all'asciutto / Conserver au sec / Mantener seco / Vor Feuchtigkeit schützen
	Contains # pouches: 1 Disposable Transfer Tool (DTT), 1 Sample Buffer Tube (SBT), 1 PIE / Contiene # buste: 1 PIE, 1 Sample Buffer Tube (SBT), 1 Disposable Transfer Tool (DTT) / Contient # sachets: Une cartouche PIE, Un tube échantillon, Un outil de transfert jetable (OTJ) / Incluye # bolsitas: 1 PIE, 1 Sample Buffer Tube (SBT), 1 Disposable Transfer Tool (DTT) / Enthält # Beutel: 1 PIE, 1 Sample Buffer Tube (SBT), 1 Disposable Transfer Tool (DTT)
	Humidity Limitation / Limitazione dell'umidità / Limite d'humidité / Limitación de humedad / Feuchtigkeitsbegrenzung
	Prescription Use Only / Per l'uso su prescrizione medica / Uniquement sur prescription / Solo Para Uso Por Receta / verschreibungspflichtig
	Do not use if package is damaged / Non utilizzare se la confezione è danneggiata / Ne pas utiliser si l'emballage est endommagé / No utilizar si el envase está dañado / Nicht verwenden, wenn die Verpackung beschädigt ist
	Keep away from sunlight / Tenere lontano dalla luce del sole / Tenir à l'écart de la lumière du soleil / Mantener alejado de la luz solar / Vor Sonnenlicht schützen
	Contains 24 Disposable Transfer Loops (DTL) / Contiene 24 loop di trasferimento monouso (DTL) / Contient 24 boucles de transfert jetables (DTL) / Contiene 24 bucles de transferencia desechables (DTL) / Enthält 24 Einwegübertragungsschleifen (DTL)
	Disposable Transfer Tool (DTT) / Disposable Transfer Tool 9DTT / Un outil de transfert jetable (OTJ) / Disposable Transfer Tool (DTT) / Disposable Transfer Tool (DTT)
	Revogene Test Device / Dispositivo Test Revogene / Dispositif de test Revogene / Dispositivo para la Prueba Revogene / Revogene-Analysegefäß
	For Emergency Use Authorization only / Solo per l'autorizzazione per l'uso di emergenza / pour autorisation d'utilisation d'urgence uniquement / para autorización de uso de emergencia solamente / nur für Notfallverwendungsautorisierung
	Disposable Transfer Loop / Loop di trasferimento monouso / Boucles de transfert jetables / Bucles de transferencia desechables / Einwegübertragungsschleifen
	Sample Buffer Tube / Sample Buffer Tube / Un tube échantillon / Sample Buffer Tube / Sample Buffer Tube
	Revogene Test Device / Dispositivo Test Revogene / Dispositif de test Revogene / Dispositivo para la Prueba Revogene / Revogene-Analysegefäß
	Swiss Authorized Representative / Mandatario svizzero / Mandataire Suisse / Representante Autorizado Suizo / Schweizer Bevollmächtigter

For technical assistance, call Technical Support Services at 800-343-3858 between the hours of 8AM and 6PM, USA Eastern Standard Time. To place an order, call Customer Service Department at 800-543-1980.

Revogene and associated logos are registered trademarks of Meridian Bioscience, Inc.
© 2020-01 Meridian Bioscience, Inc.
Made in United States

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Revogene e I loghi associate sono marchi commerciali di Meridian Bioscience, Inc.
Revogene et les logos connexes sont des marques déposées de Meridian Bioscience, Inc.
© 2020-001 Meridian Bioscience, Inc.
Revogene así como los logotipos asociados son marcas comerciales de Meridian Bioscience, Inc.
©2020-001 Meridian Bioscience, Inc.
Revogene und die damit verbundenen Logos sind Marken von Meridian Bioscience, Inc.