

revogene® check

Verification tool for use with the Revogene®



www.meridianbioscience.com/pi
T: 1-800-343-3858

IVD

REF 610220

Quick Start Guide

This is provided as an overview.
Please consult the package insert
for complete instructions.

Preparing the Revogene® Check

1. Remove one (1) pouch from the box. Open each side of the pouch and remove PIE and Sample Buffer Tube (SBT). Use the PIE within one (1) hour after opening the pouch.
2. Use the Disposable Transfer Tool (DTT) provided in the right side of the pouch to aspirate Sample Buffer (SB) to a level anywhere between the two (2) marks on the DTT. Discharge the entire SB volume into the sample loading chamber of the PIE. Close the cap of the PIE tightly.

Running the Revogene® Check

1. Log in by entering the <Username> and <Password> and tap <Login>. The main menu will appear automatically.
2. Select <Tools> from the main menu.
3. Select <Revogene Check> from the tools menu.
4. Scan the Master barcode located on the Quick Start Guide.
5. Scan the provided SBT and PIE.
6. Tap <Next>.
7. Insert the PIE into the Revogene at any position of the carousel.
8. Insert MOCK PIEs in the carousel remaining positions.
9. Scan the retention ring and place it on the carousel.
10. Close the instrument lid with both hands. Initiate Revogene Check run by tapping <Start>. The test will run for approximately 70 minutes.

Revogene® Check Results

Overall Result

Pass

Detailed Result By Module

Performance Module Tested		Result
PCR System	Optical Detection	PASS
Interface Communications		PASS

Revogene verification completed.

The instrument calibration is extended for an additional 12 months. Resume testing of patient specimens.

Overall Result

Calibration Required

Detailed Result By Module

Performance Module Tested		Result
PCR System	Optical Detection	Calibration Required
Interface Communications		PASS

Revogene verification indicates that calibration is required.

If this result is obtained with the first PIE, repeat the process with the second PIE in the Revogene Check kit box.

If the second PIE indicates that calibration is required, contact Technical Support at: 1-800-343-3858

Revogene® Check Master Barcode



Meridian Bioscience Inc.
3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513-271-3700

Orders/Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Email: MBI-TechService@meridianbioscience.com
meridianbioscience.com