



TEST PROCEDURE *How to perform the test*

1. Add 2 drops of serum to both lower ports. Incubate for 2 minutes at 22-25 C.

2. Add 3 drops of Conjugate to both lower ports. Incubate for 2 minutes at 22-25 C.

3. Add 3 drops of Wash Buffer to both upper ports. Allow Wash Buffer to absorb.

4. Add 2 drops of Substrate to both upper ports. Incubate for 5 minutes at 22-25 C. Interpret visually.

INTERPRETATION OF RESULTS

REACTIVE

NONREACTIVE

INVALID

INVALID

READ ONLY UPPER REACTION PORTS TO INTERPRET RESULTS

This illustration is representative of the current Package Insert at the time of publication. Please refer to the most current version of the package insert for complete instructions.

USA/Corporate Office
3471 River Hills Drive
Cincinnati, Ohio 45244
Telephone: 513.271.3700
www.meridianbioscience.com

Orders / Customer Service: 1.800.543.1980
Technical Support: 1.800.343.3858
Information Fax: 513.272.5432
Ordering Fax: 513.271.0124



© 2009 SN11232, REV. 01/10