


immunocard[®]

H.pylori

TEST PROCEDURE

How to perform the test

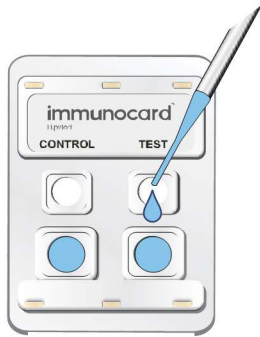

1. Add 2 drops of serum or plasma to each lower port and wait 1 minute.



2. Add 2 drops of Enzyme Conjugate followed by 2 drops of Wash Buffer to both upper ports. Allow the Enzyme Conjugate to completely absorb prior to the addition of Wash Buffer.



3. After all the Wash Buffer has soaked in entirely, add 2 drops of Substrate to both upper ports. Start timer for 5 minutes.

Interpretation of Results



Figure 1.
Positive
Result



Figure 2.
Negative
Result



Figure 3.
Invalid
Result



Figure 4.
Invalid
Result



This illustration is representative of the current Package Insert at the time of publication. Please refer to the most current version of the Package Insert for complete instructions.

USA/CORPORATE OFFICE

3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513-271-3700

©2018 SNT1234 REV. 08/18

MERIDIAN BIOSCIENCE EUROPE

Belgium/Luxembourg Tel: +32 (0) 6789 5959
France Tel: +33 (0) 1 4256 0440
Germany Tel: +49 (0)3371 60 222 31
Italy Tel: +39 0331 43 3636
Netherlands Tel: +31 (0) 411 62 11 66
United Kingdom Tel: +44 (0) 20 8453 7970

Orders/Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Information Fax: 513-272-5432
Ordering Fax: 513-271-0124
meridianbioscience.com

meridian BIOSCIENCE™