



TEST PROCEDURE
How to perform the test

immunocard STAT![®]


Rotavirus

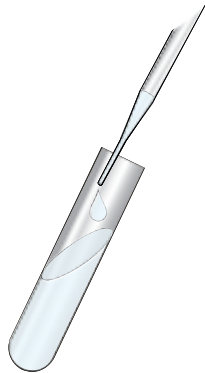

1. Add 350 μ L of Sample Diluent to a test tube.

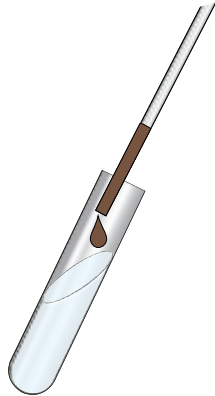


2. Add 25 μ L of Stool Sample to the test tube. Mix Well.


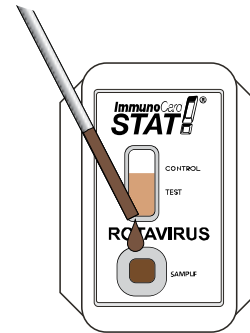


3. Add 150 μ L of diluted sample to sample port. Incubate 10 minutes at 21 - 25 C. Interpret Results.


Interpretation of Results

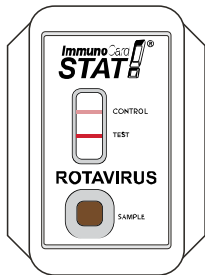


POSITIVE

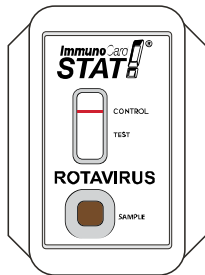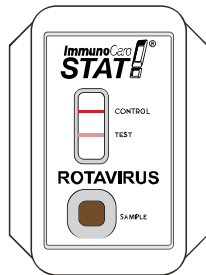

NEGATIVE

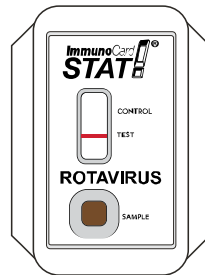

INVALID

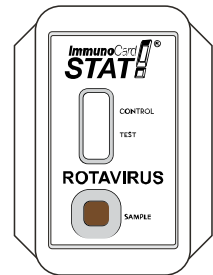

This illustration is representative of the current Package Insert at the time of publication. Please refer to the most current version of the Package Insert for complete instructions.

USA/CORPORATE OFFICE

3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513-271-3700

©2018 SN11241 REV. 10/18

MERIDIAN BIOSCIENCE EUROPE

Belgium/Luxembourg	Tel: +32 (0) 6789 5959
France	Tel: +33 (0) 1 4256 0440
Germany	Tel: +49 (0)3371 60 222 31
Italy	Tel: +39 0331 43 3636
Netherlands	Tel: +31 (0) 411 62 11 66
United Kingdom	Tel: +44 (0) 20 8453 7970

Orders/Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Information Fax: 513-272-5432
Ordering Fax: 513-271-0124
meridianbioscience.com

meridian BIOSCIENCE™