


ImmunoCard STAT! ROTAVIRUS

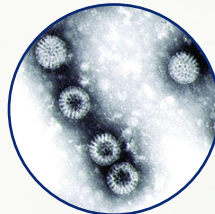

Knowing matters. So when you need to know NOW, trust Meridian for the solution.


ImmunoCard STAT!® Rotavirus provides accurate results for a differential diagnosis in just minutes. In fact, more laboratories trust Meridian for rapid detection of Rotavirus than any other test on the market.

- Rapid results in approximately 10 minutes
- Simple procedure that requires no filtration
- Trusted, accurate performance: Sensitivity 93.1%
Specificity 95.8%
- Clearly defined test bands for easy interpretation
- Controls included


Test Procedure



1. Add 350 μ L of Sample Diluent to a test tube.



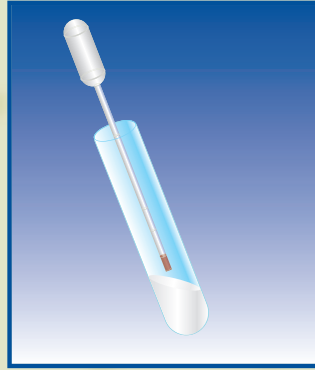


2. Add 25 μ L of stool sample to the test tube. Mix well.

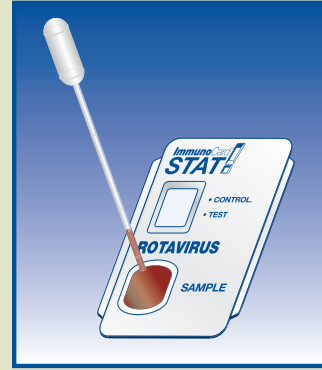

3. Add 150 μ L of diluted sample to sample port. Incubate 10 minutes at 21 -25 C. Interpret results.

Interpretation of Results

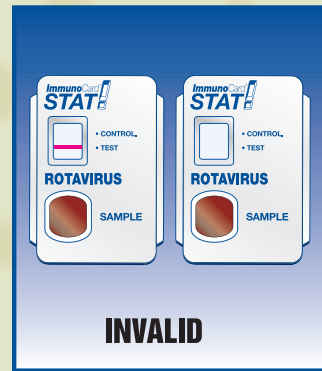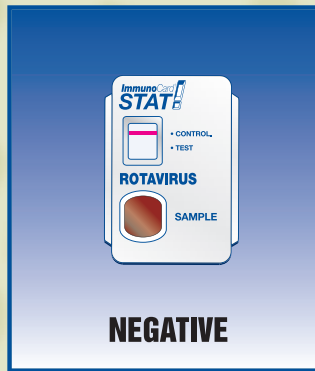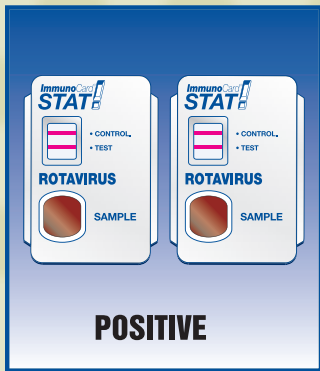

*This chart does not contain complete instructions for use. For further information, please refer to the package insert.

* The following illustration depicts approximate sample size when testing solid, semi-solid, or non-pipetable stools.



Ordering Information

ImmunoCard STAT![®] Rotavirus

Test Size

30 Tests

Catalog #

750030

Other Meridian Products

Premier Rotaclone[®]

Premier Adenoclone[®] 40/41

Premier Adenoclone[®] GSA

Test Size

48 determinations

48 determinations

48 determinations

Catalog #

696004

696006

696007



USA/Corporate Office:
3471 River Hills Drive
Cincinnati, Ohio 45244
Telephone: 513.271.3700
Orders/Customer Service: 1.800.543.1980
Technical Support: 1.800.343.3858
Information Fax: 513.272.5432
Ordering Fax: 513.271.0124
www.meridianbioscience.com

Belgium / France / Italy / The Netherlands

