


immunocard STAT![®]


HpSA[®]

TEST PROCEDURE


How to perform the test

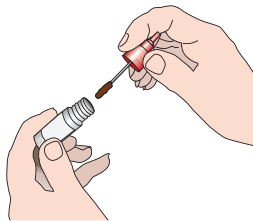

1. Using the applicator stick of the diluent vial, transfer a small portion (5-6 mm diameter) of stool specimen into the sample diluent. Vortex 15 seconds.

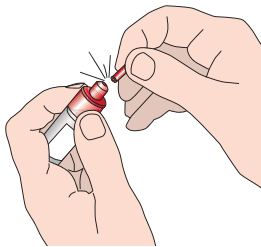


2. Hold the vial vertically and gently tap on benchtop. Break the tip from vial using absorbent paper.

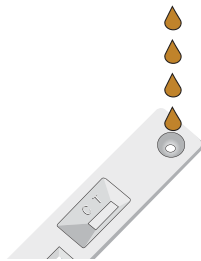


3. Dispense 4 drops into the round window of the device. After 5 minutes, read result within 1 minute.

Interpretation of Results

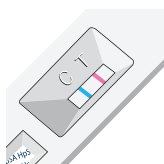

POSITIVE

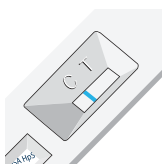

NEGATIVE



INVALID



This chart does not contain complete instructions for use. For further information, please refer to the package insert.

USA/CORPORATE OFFICE

3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513-271-3700

MERIDIAN BIOSCIENCE EUROPE

Belgium/Luxembourg Tel: +32 (0) 6789 5959
France Tel: +33 (0) 1 4256 0440
Germany Tel: +49 (0)3371 60 222 31
Italy Tel: +39 0331 43 3636
Netherlands Tel: +31 (0) 411 62 11 66
United Kingdom Tel: +44 (0) 20 8453 7970

Orders/Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Information Fax: 513-272-5432
Ordering Fax: 513-271-0124
meridianbioscience.com

meridian BIOSCIENCE[®]