## POSITIVE CONTROL

External Positive Control Set for ImmunoCard STAT! CGE  
(CRYPTOSPORIDIUM / GIARDIA / ENTAMOEBIA)

**REF** 751403

**IVD** In vitro diagnostic medical device

### INTENDED USE

**ImmunoCard STAT! CGE Positive Control** is an external control to be used with the ImmunoCard STAT! CGE kit (Catalog# 751420), as part of a routine quality control program.

The Positive Control reagent contains inactivated antigen of the parasites *Cryptosporidium parvum*, *Giardia lamblia*, and *Entamoeba histolytica* in a buffered solution. The Positive Control is ready to use. Use as supplied.

### SUMMARY AND EXPLANATION OF THE TEST

Quality control testing is performed to detect factors such as reagent deterioration, adverse environmental or test conditions, or variance in operator performance that can cause test errors. External control reagents, such as ImmunoCard STAT! CGE Positive Control, are reagents that are not built into the test system, but are sampled and tested in the same manner as patient specimens.

### BIOLOGICAL PRINCIPLES

**ImmunoCard STAT! CGE Positive Control** is used in conjunction with the Sample Diluent/Negative Control provided with each ImmunoCard STAT! CGE kit to create a known positive and negative control panel. The frequency with which a laboratory performs external controls will be affected by:

1. The proficiency level of the laboratory,
2. The laboratory's own internal requirements,
3. The requirements of the laboratory's accrediting agencies,
4. The number of new and different operators performing the test,
5. Whether a new kit lot is being added to testing,
6. Whether the kit lot is from a different shipment,
7. Whether deviations from the manufacturer's stated storage or handling conditions have occurred.

When unacceptable quality control test results are obtained, all test results should be considered invalid. QC test failures can mean that one of the reagents or components is no longer reactive at the time of use, the test was not performed correctly or that reagents or sample were not added.

### REAGENTS/MATERIALS PROVIDED

**The maximum number of test obtained from this test kit is listed on the outer box. ImmunoCard STAT! CGE Positive Control:** buffered solution containing inactivated antigen of the parasites *Cryptosporidium parvum*, *Giardia lamblia* and *Entamoeba histolytica*.

The stability of the positive control is 24 months if stored at 2-8 C. Positive control has been evaluated against the most risk infectious agents (Hepatitis, HIV) leading to a negative result. The reagent is ready to use

### MATERIALS NOT PROVIDED

1. ImmunoCard STAT! CGE Kit (Catalogue # 751420)
2. Timer

### PRECAUTIONS

1. All reagents are for in vitro diagnostic use only.
2. This is a quality control reagent and is used to evaluate the performance of ImmunoCard STAT! CGE. It is not directly used to test patient samples.
3. Store this reagent at 2-8 C when not in use. Do not freeze.
4. This reagent can be used with any ImmunoCard STAT! CGE kit lot number.

### HAZARD and PRECAUTIONARY STATEMENTS

There are no known hazards associated with this product.

### QUALITY CONTROL TEST PROCEDURE

**The test should be performed per applicable local, state, or federal regulations or accrediting agencies.**

1. Bring ImmunoCard STAT! CGE Positive Control and all test kit components to room temperature (19-27 C) before testing. Vortex for approximately 10 seconds.
2. Use one ImmunoCard STAT! CGE card for each positive and negative control reagent to be tested. Remove each Test Card from its foil pouch. Label each card with the name of the control to be tested.
3. Using a new pipette tip, transfer 125 µL of the Diluent/Negative Control to the sample area of an appropriately labelled card.
4. Using a new pipette tip, transfer 125 µL of the Positive Control to the sample area of an appropriately labelled card.
5. Incubate the test for 10 minutes at 19-27 C.
6. At the end of testing, read results as described in the section INTERPRETATION OF RESULTS of ImmunoCard STAT! CGE insert.

**If the expected control reactions are not observed, repeat the control tests as the first step in determining the root cause of the failure. If control failures are repeated please contact Meridian's Technical Services Department at 1-800-343-3858 (US) or your local distributor.**

### LIMITATIONS OF THE PROCEDURE

The ImmunoCard STAT! CGE Positive Control is designed for use only with the ImmunoCard STAT! CGE Rapid Assay. The validity of the reactions they produce in other similar tests cannot be guaranteed.

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