

Procedure Card



TEST PROCEDURE

How to perform the test

1. Using the dropper with the Sample Diluent/Negative Control, add 1 mL of Sample Diluent to a test tube.

2. Add a small portion of approximately 5-6 mm size (50 mg) or 100 µL stool using appropriate applicator or pipette. Vortex 15 seconds.

3. Wait 3-5 minutes until the solid particles settle at the bottom of the tube or centrifuge.

4. Add 125 µL of the prepared specimen to the sample port of the device and read the results at 10 minutes.

RESULTS

— Control
— Entamoeba
— Giardia
— Cryptosporidium

Negative Positives

This illustration is representative of the current Package Insert at the time of publication. Please refer to the most current version of the Package Insert for complete instructions.

© 2015 SN751420 REV. 06/15

USA/CORPORATE OFFICE
 3471 River Hills Drive
 Cincinnati, Ohio 45244 USA
 Telephone: 513.271.3700
 Orders / Customer Service: 1.800.543.1980
 Technical Support: 1.800.343.3858
 Information Fax: 513.272.5432
 Ordering Fax: 513.271.0124
www.meridianbioscience.com

MERIDIAN BIOSCIENCE EUROPE
 Belgium/Luxemburg Tel: +32 (0) 67 89 59 59
 France Tel: +33 (0) 1 42 56 04 40
 Germany Tel: +49 (0) 3371 681 500
 Italy Tel: +39 0331 43 3636
 Netherlands Tel: +31 (0) 411 62 11 66
 United Kingdom Tel: +44 (0) 20 8453 7970

AUSTRALIA
 Bioline (Aust) Pty Ltd
 Tel: 1 800 963 120
 Email: info.aust@bioline.com



For other countries, please visit: www.meridianbioscience.eu/distributors