



TEST PROCEDURE

How to perform the test

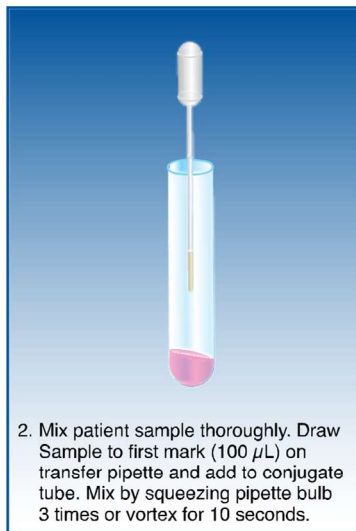

RESULTS

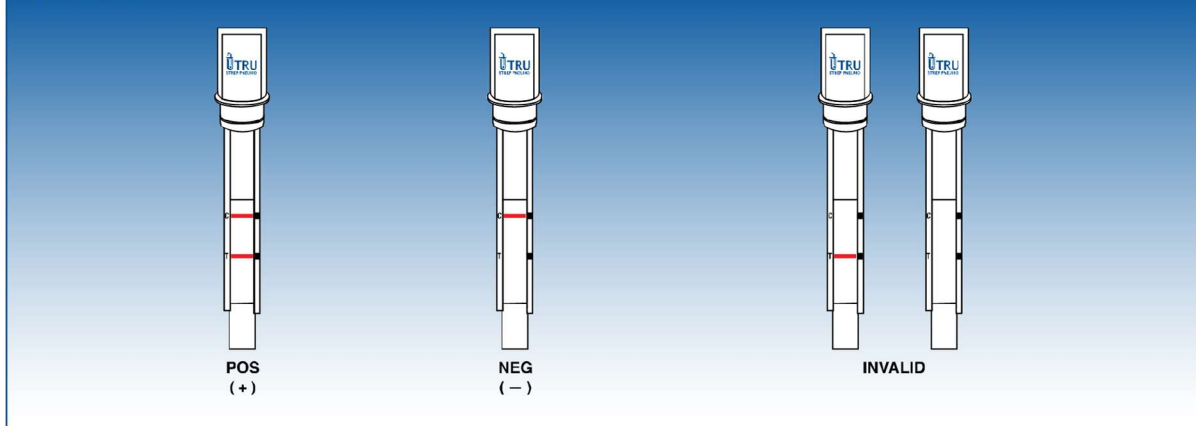

EXTERNAL CONTROLS TEST:

1. Positive Control: Add exactly 5 drops of the Positive Control reagent to the Conjugate Tube marked for the Positive Control.
2. Sample Diluent/Negative Control: Add 200 µL (second mark) to the Conjugate Tube marked for the Negative Control.
3. Vortex or swirl the contents of the tubes for 10 seconds.
4. Insert the narrow end of a Test Strip to each Conjugate Tube and firmly press down on the caps to close each tube. Incubate both tubes at 19-27 C for 20 minutes.



Please refer to the most current version of the package insert for complete instructions.

© 2014 SN11133 REV. 01/15

USA/CORPORATE OFFICE
3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513.271.3700

Orders/Customer Service: 1.800.543.1980
Technical Support: 1.800.343.3858
Information Fax: 513.272.5432
Ordering Fax: 513.271.0124
www.meridianbioscience.com

MERIDIAN BIOSCIENCE EUROPE
Belgium/Luxembourg Tel: +32 (0) 67 89 59 59
France Tel: +33 (0) 1 42 56 04 40
Italy Tel: +39 0331 43 3636
Netherlands Tel: +31 (0) 411 62 11 66

 **Meridian**
Bioscience® Europe
Inspired Science. Trusted Solutions.®