


TEST PROCEDURE


How to perform the test

1

Note: Thoroughly mix stool specimen by vortexing for approximately 5 seconds or inverting (e.g., 3-4 times). Remove yellow cap from the Aioprep device and set aside.

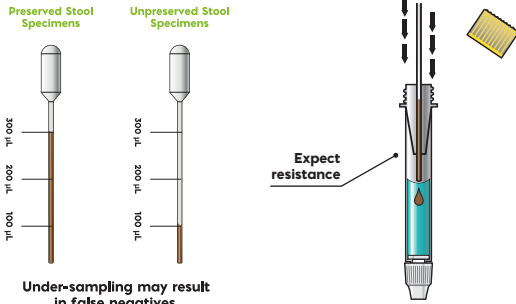

2

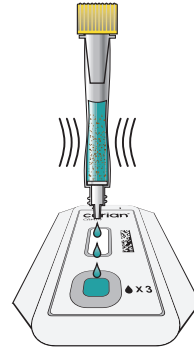

Pipetable Sample:

- **Preserved Stool Specimens:** using the transfer pipette provided, transfer **300 µL** of preserved stool (**third mark from pipette tip**) directly through Aioprep device. Recap Aioprep.
- **Unpreserved, Liquid / Semi-solid Stool Specimens:** using the transfer pipette provided, transfer **100 µL** of unpreserved stool (**first mark from pipette tip**) directly through Aioprep device. Recap Aioprep.
- **Do not invert or shake Aioprep.** Mix the sample in the Aioprep sample diluent by vortexing for approximately 5 seconds. Sample must be thoroughly suspended in the Aioprep sample diluent.

Non-pipetable Sample:

Use of brush needed. Refer to Package Insert for full instructions.

- After mixing the sample, remove 1 test card from its foil pouch.
- Remove white tip cap from bottom of the Aioprep and discard.
- Holding vertically, squeeze the center of the barrel to add 3 drops into the "SAMPLE" port of the test card.
- Discard Aioprep device immediately.
- Start the test read by navigating to the "TEST" menu. Follow the on-screen instructions.

3



OR



- **Analyze Now:** Incubate the test card at 19-27 C on the bench top for 20 minutes. Insert test card into the drawer of the Curian Analyzer and read test within 2 minutes of the end of incubation.
- **Incubate and Analyze:** Insert the test card immediately into the analyzer and the analyzer will time the incubation for 20 minutes.
- The analyzer will read the test card and automatically report the test result.



This illustration is representative of the current and complete instructions for use at the time of publication. Please refer to the most current complete versions of the assay and instrument instructions. www.meridianbioscience.com/pi

USA/CORPORATE OFFICE

Meridian Bioscience
3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513-271-3700

Orders/Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Information Fax: 513-272-5432
Ordering Fax: 513-271-0124
meridianbioscience.com