


FLU A&B


Nasopharyngeal Wash/Aspirate Sample Procedure


TEST DEVICE

Extraction Well

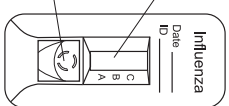


Result Window

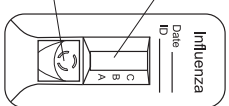

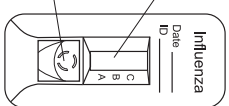


PROCEDURE

1 Draw nasal wash or nasal aspirate sample to the **first (lowest) mark** of the graduated transfer pipette.

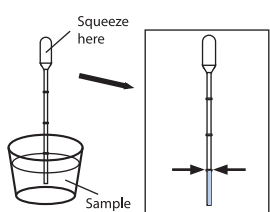


2 Dispense the entire sample in the transfer pipette into the Extraction Well of the test device.

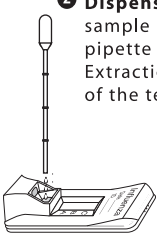


3 Remove the cap from the Extraction Reagent bottle.

Using a new transfer pipette, draw Extraction Reagent Solution to the **first (lowest) mark**.




4 Dispense all of the solution in the transfer pipette into the Extraction Well of the test device.

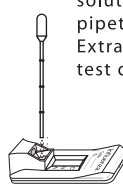


5 Let stand 1 minute. Re-cap the Extraction Reagent bottle.


6 Raise the device upright and let stand 1-2 seconds.



7 Gently tap device to ensure the liquid flows into the hole.

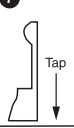


Then, lay the device back down.

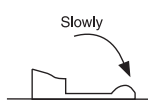


Start timing.

8 Read test results at 10-15 minutes. Confirm negative results at 15 minutes.


INTERPRETATION OF RESULTS

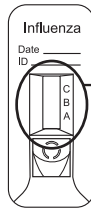


Any visible A and/or B Line(s) with C Line is **POSITIVE**.

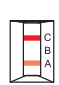

A line:
Influenza type A

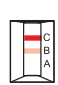


B line:
Influenza type B

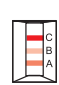


A & B lines:
Influenza type A & B

C Line only
NEGATIVE

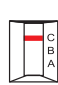

No C Line
INVALID

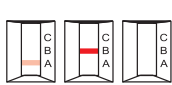

Repeat with new sample and device.



This illustration is representative of the current Package Insert at the time of publication. Please refer to the most current version of the Package Insert for complete instructions.

MF Manufactured for:

Meridian Bioscience
3471 River Hills Dr.
Cincinnati, OH 45244 USA
Tel: 1-513-271-3700

To Place An Order: 1-800-543-1980

For Technical Support: 1-800-343-3858

www.meridianbioscience.com



EC REP

MT Promedct Consulting GmbH
Ernst-Heckel-Straße 7
66386 St. Ingbert, Germany
Tel: +49-68 94-58 10 20



Manufactured by:

Princeton BioMeditech Corp.

4242 U.S. Hwy 1
Monmouth Junction, NJ 08852 USA
Tel: 1-732-274-1000
www.pbmc.com