REF 901312, 981312

IVD

Rx Only

INTENDED USE
Para-Pak/Para-Pak ULTRA EcoFix is a convenient one-vial system for the routine collection, transportation, preservation, and examination of stool specimens for intestinal parasites. Kit systems are designed for easy use by individuals not trained in microbiological procedures and afford an excellent means of minimizing the adverse effects of delay in specimen transportation. Both concentration and permanent stain may be performed from the EcoFix preserved specimen. The fixative affords an excellent means of minimizing the adverse effects of delay in specimen transport.

SUMMARY AND EXPLANATION OF THE TEST
Diagnosis of intestinal parasitic disease is confirmed by recovery and identification of helminth eggs and larvae, or protozoan trophozoites and cysts in the clinical parasitology laboratory. Timely collection and transportation of "fresh" stool specimens to the laboratory cannot always be assured. Workload conditions and priorities in clinical laboratories frequently do not permit immediate examination of "fresh" specimens. Procedures such as incubation, refrigeration, or freezing will not guarantee the recovery of all diagnostic stages of all parasites.^{1-5, 8, 9}

Proper use of Para-Pak/Para-Pak ULTRA EcoFix thus assures the parasitologist that diagnostic stages of intestinal parasites will be preserved if present in fecal material. Para-Pak/Para-Pak ULTRA EcoFix was developed as a formalin-free and mercury-free fixative in response to toxic chemical disposal problems and monitoring regulations.¹⁰

BIOLOGICAL PRINCIPLES
EcoFix provides a multi-purpose mercury and formalin-free fixative/preservative suitable for a variety of parasitological procedures. EcoFix preserved specimens may be concentrated and used to prepare permanently stained slides.

REAGENTS/MATERIALS PROVIDED
The maximum number of tests obtained from this test kit is listed on the outer box.
Each vial contains 15 mL EcoFix with a built-in collection spoon. EcoFix vials are also available in case quantities. Simple directions for patients and nursing personnel are also provided in 10 languages.

Catalog #901312
EcoFix (item #9014)

Catalog #981312
EcoFix (item #9014)
Key, Natural (item #14468)
Surfactant (item #9601-02)

- MATERIALS NOT PROVIDED**
- 10% Formalin
 - Ethyl acetate (or Hemo-De)
 - Physiological Saline
 - Lugol's Iodine
 - Cotton Tipped Applicator Sticks
 - Concentration System (Para-Pak® CON-Trate® or Macro-CON®)
 - Transfer Pipettes
 - Microscope Slides and Cover Slips
 - Centrifuge
 - Microscope

- PRECAUTIONS**
- All reagents are for in vitro diagnostic use only.
 - Avoid contact of EcoFix solution with the skin and eyes. Avoid breathing fumes. Should contact occur, flush with running water. If irritation develops, contact a physician.
 - EcoFix solution is poisonous. If ingested, call the local poison center or a physician immediately.
 - Due to the infectious nature of unpreserved stools, use of gloves, care, and handwashing should be employed when the specimen is obtained and manipulated.
 - Patient specimens may contain HIV or other infectious agents and should be handled by properly trained personnel and disposed of as potential biohazards. Wear disposable gloves while handling patient specimens and performing test procedures.
 - EcoFix is flammable.
 - Any serious incident that has occurred in relation to the device should be reported to Meridian Bioscience, Inc., 3471 River Hills Drive, Cincinnati, Ohio 45244 USA or Technical Support Center 800.343-3858 and competent authority of the EU Member State in which the clinician and/or patient is established.
 - IMPORTANT:** See SDS for additional safety and hazard information.

HAZARD and PRECAUTIONARY STATEMENTS

 <p>Para-Pak and Para-Pak ULTRA EcoFix</p>	<p>Signal Word Danger</p> <p>Hazard statements H303 - May be harmful if swallowed H313 - May be harmful in contact with skin H318 - Causes serious eye damage H331 - Toxic if inhaled H411 - Toxic to aquatic life with long lasting effects H225 - Highly flammable liquid and vapor Contains Zinc sulfate, Acetic acid, Methyl alcohol</p> <p>Precautionary Statements - EU (§28, 1272/2008) P321 - See SDS Section 4 or Section 11 for specific medical treatment information P403 + P233 - Store in a well-ventilated place. Keep container tightly closed P280 - Wear eye protection/ face protection P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing P310 - Immediately call a POISON CENTER or doctor/ physician P210 - Keep away from heat/sparks/open flames/hot surfaces. - No smoking P370 + P378 - In case of fire: Use dry sand, dry chemical or alcohol-resistant foam for extinction</p>
---	---

 <p>Reagent A (MucoPenX), CON-Trate Reagent A and Macro-CON Surfactant</p>	<p>Signal word Warning</p> <p>Hazard Statements H302 - Harmful if swallowed Contains Polyethylene glycol 1,1,3,3-tetramethylbutylphenyl ether</p>
---	---

SAFETY AND DISPOSAL
EcoFix may be considered a hazardous waste in some areas. Consult local authorities for the regulations on disposal in your area.

SHELF LIFE AND STORAGE
Expiration dating for Para-Pak/Para-Pak ULTRA EcoFix is indicated on the outer packaging. Store at room temperature (15-30 C). Excessive heat and cold should be avoided.

SPECIMEN COLLECTION AND PREPARATION

1. The patient should be cautioned against the use of antacids, barium, bismuth, antidiarrheal medication, or oily laxatives prior to collection of the specimen.
2. To assure recovery of parasitic elements which are passed intermittently and in fluctuating numbers, three specimens spaced a few days apart must be examined. In the case of hospitalized patients it is suggested that all fecal passages be collected for a designated length of time to avoid prolonging the hospital stay.^{1,4}
3. The specimen is ideally passed into a bedpan but must not be contaminated with urine. Alternatively, a large plastic bag or "saran wrap" may be placed over the toilet seat opening and the specimen passed into the bag. A thoroughly cleaned and dried milk carton, cut so as to remove the upper two thirds of the carton may also be used. It will be easier to collect the specimen if the water supply to the toilet is shut off and water drained from the bowl by flushing twice.
4. An appropriate (i.e. bloody, mucoid, watery) area of stool should be selected and sampled with the collection spoons provided in the caps of the containers. Sufficient stool is added to each container to bring the liquid level up to the "Fill to Here" line. This will result in approximately 5 mL of sample. To insure adequate sampling of a formed stool, material should be removed from the sides, ends, and middle of the bolus.
5. Agitate each specimen with the spoon along the sides of the container, tighten the cap and shake firmly to insure that the specimen is adequately mixed. When mixing is completed the specimen should appear homogenous.
6. Return the vials to their container, seal the container, and label appropriately.

TEST PROCEDURE

The Para-Pak EcoFix System lends itself to a wide variety of procedures in common use. The following discussion is not exhaustive and alternatives may be found. While variations may exist from lab to lab, a thorough examination should include the following steps:

1. Gross examination: Record the presence of blood, worms, mucus, or proglottids.
2. Direct microscopic examination from the EcoFix preserved specimen:
 - a. Place a clean glass slide on a sheet of newsprint.
 - b. Add a drop of saline to the slide. **(NOTE: If utilizing the direct exam, iodine cannot be used to stain the prep.)**
 - c. Add a representative sample of EcoFix preserved specimen to the drop of saline and mix thoroughly with the collecting spoon. The newsprint must be just legible through the slide.
 - d. Place a double width coverslip on the suspension and examine immediately.
3. Permanent slide and concentration procedure using the Para-Pak ULTRA EcoFix:
 - a. **Permanent Slide Preparation:**
 - (1) Using the provided key, remove the protective cap from the vial.
 - (2) Snap and twist off the inner tip.
 - (3) If the specimen appears to be mucoid, add 10 drops of surfactant through the tip of the vial. Swirl gently to facilitate mixing.
 - (4) The vial can be used with a 15 mL or 50 mL conical tube. Insert the tube into the top of the Para-Pak ULTRA EcoFix vial. Invert and filter (by squeezing the vial) at least 3-5 mLs of specimen through the filtration device which is contained in the vial. (If desired, the entire specimen may be filtered.)
 - (5) While holding the vial and tube at a 30 degree angle, remove the vial from the conical centrifuge tube.
 - (6) Centrifuge at 500-1000 xg for 10 minutes (1800-2200 rpm for most table top centrifuges).
 - (7) Decant the supernatant fluid. Approximately 0.5 to 1.0 mL of sediment should remain. If necessary, remove sediment or add more filtered specimen and repeat steps 4-6.
 - (8) Mix the sediment in the EcoFix which drains back from the walls of the tubes. A small drop of EcoFix may be added if necessary and prepare a smear for permanent staining as follows:^{6,7}
 - (a) Add a small amount of the suspended sediment to the slide.
 - (b) Spread the mixture over the slide to produce a film which varies in thickness.
 - (c) Allow the slide to dry overnight at room temperature or for several hours in a 37 C incubator or slide warmer. Accelerated drying may cause some morphological distortion. The slides must be dried thoroughly to avoid washing the film off during staining.
 - (d) Once slides prepared in EcoFix have dried completely, **staining may begin by placing the slides directly into the Trichrome stain.** (NOTE: EcoStain may also be used - refer to package insert Product Catalog # 801400). (NOTE: Iron hematoxylin stain may also be used).
 - (e) Staining Procedure*

1) Trichrome Stain	6-8 minutes
2) 90% Alcohol Acidified	2 quick dips
3) 95% Alcohol	Dip twice
4) 95% Alcohol	5 minutes
5) 100% Ethanol or Carbol-xylene	10 minutes
6) Xylene	10 minutes
7) Mount with coverslip using the mounting medium of choice.	

*Manual of Clinical Microbiology ASM publication and Para-Pak Trichrome Stain Package Insert
 - b. **Alternate procedure for preparation of permanent slide:**
 - (1) Pour some of the EcoFix-fixed material onto a paper towel and allow to stand for 3 minutes. This will absorb out excess EcoFix and is considered critical to obtain the best possible staining.
 - (2) Using an applicator stick or brush, spread (avoid smearing) some of the specimen from the paper towel onto 1 or more clean glass slides. For best adherence, spread the material to the edge of the slide.
 - (3) The slides are dried overnight at room temperature or for several hours in a 37 C incubator or slide warmer. Accelerated drying may cause some morphological distortion. The slides must be dried thoroughly to avoid washing the film off during staining.
 - (4) Continue with staining method (see step (8)(d) above).
 - c. **Concentration procedure:**
 - (1) To the remaining sediment from Step a.(7) above, add 9-10 mL of 10% formalin (or saline), mix and allow to stand for 10 minutes.^{1,3}
 - (2) Add 3 mL of ethyl acetate (or Hemo-De), then stopper and shake the tube vigorously for at least 30 seconds. Carefully remove the stopper.
 - (3) Centrifuge for 10 minutes at 500-1000 xg (1800-2200 rpm for most table top centrifuges).
 - (4) Four layers will be apparent:
 - (a) Top layer: ethyl acetate (or Hemo-De)
 - (b) Second layer: plug of debris
 - (c) Third layer: formalin (or saline)
 - (d) Bottom layer: sediment
 - (5) After ringing the plug of debris from the sides of the tube with an applicator stick, carefully decant the top 3 layers. While keeping the tube inverted, a cotton swab should be used to remove excess ethyl acetate (or Hemo-De) and debris sticking to the sides of the tube. This is particularly important for obtaining suitable results with ethyl acetate (or Hemo-De) and avoids solvent bubbles in the wet mount.
 - (6) Add a few drops of physiological saline or 10% formalin to re-suspend the remaining sediment. NOTE: Recommend re-suspension in 10% formalin if any delay in reading the concentrate is anticipated. If the resulting slides are too dense (newsprint should be legible through them) more saline or 10% formalin may be added.
 - (7) Iodine and saline mounts are suggested for microscopic examination.
NOTE: If the pellet in step 6 contains a large amount of debris, a wash step may be performed. Re-float the sediment in 7 mL saline, shake, and re-centrifuge at 500 xg. Pour off supernatant and continue with steps 6 and 7.
4. Permanent slide and concentration procedure for Para-Pak EcoFix using the Para-Pak Macro-CON.
 - a. Thoroughly mix the material in the Para-Pak EcoFix specimen collection vial until it appears homogeneous. Follow the Para-Pak Macro-CON package insert, Specimen Processing, steps 1 through 8. Insert the following procedure between steps 8 and 9 of the Macro-CON Specimen Processing section.
 - (1) Place provided screw cap on the conical centrifuge tube and centrifuge for 10 minutes at 500-1000 xg (1800-2200 rpm for most tabletop centrifuges).
 - (2) Carefully pour off the supernatant fluid.
 - (3) Mix the sediment with an applicator stick in the EcoFix which drains back from the walls of the tube. A small drop of EcoFix may be added if necessary.
 - (4) Prepare a smear for permanent staining by following step 8 in Section 3.a. above.
 - b. Use the remaining sediment to follow Specimen Processing Steps 9-17 in the Para-Pak Macro-CON package insert. NOTE: Centrifuge for 10 minutes instead of 2 minutes in Step 12. Recommend resuspension of final concentration pellet in 10% formalin if any delay in reading the concentrate is anticipated.
5. Permanent Slide and Concentration Procedure using the Para-Pak CON-Trate System:
 - a. Thoroughly mix the material in the EcoFix vial until it appears homogenous. Follow the Para-Pak CON-Trate System package insert, Specimen Processing Steps 4.a. through e. NOTE: DO NOT add 10 mL saline in step 4.e. Centrifuge for 10 minutes in steps 4.e. and 4.i. instead of 2 minutes at 500-1000 xg. Prepare permanent slide by following Step 8 in Section 3.a. above. Continue with Steps 4.g through k. Recommend re-suspension of final concentration pellet in 10% formalin if any delay in reading the concentrate is anticipated.
6. If either the Para-Pak Macro-CON or CON-Trate Systems are unavailable, the following procedure for permanent slides and concentration may be utilized:
 - a. **Permanent Slide Preparation:**
 - (1) Thoroughly mix the material in the EcoFix tube until it appears homogenous.
 - (2) Strain approximately 2-3 mL (amount will vary in direct proportion to the density of the suspension) of the suspended material through 1 layer of narrow mesh or 2 layers of wide mesh gauze into a suitable centrifuge tube.
 - (3) Centrifuge 10 minutes at 500 xg.
 - (4) Decant the supernatant fluid. Approximately 0.5 to 1.0 mL of sediment should remain. If necessary, remove sediment or add more strained suspension and repeat Steps 3 and 4.
 - (5) Mix the sediment and prepare the permanent smear as described above, Step 8 in section 3.a. above.
 - b. Use the remaining sediment for concentration. Follow steps 3.c. 1-7 as described above.

QUALITY CONTROL

This test should be performed per applicable local, state, or federal regulations or accrediting agencies.

1. The Para-Pak/Para-Pak ULTRA EcoFix vial should contain approximately 15 mL of fluid to insure 1:3 stool to preservative ratio.
2. When an EcoFix fixed film of stock trophozoite or human buffy coat is stained, the organisms or cells should appear well fixed and defined.

If the expected control reactions are not observed, repeat the control tests as the first step in determining the root cause of the failure. If control failures are repeated please contact Meridian's Technical Services Department at 1-800-343-3858 (US) or your local distributor.

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