INSTRUCTIONS FOR USE


Preparation Guidelines for Home Collection

The bowel movement should be passed in the disposable sample collection paper.

- Do Not urinate on the bowel movement or in the collection paper.
- Do Not pass the bowel movement in the toilet.
- Do Not let collection paper touch the water.


Step 1

Remove contents from package. It contains all necessary items for proper collection of a bowel movement (stool).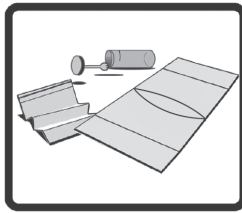


Step 2

Fill out the patient information on the vial.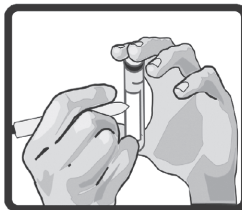


Step 3

Open collection paper carefully in the direction of the arrows. Attach tape to top of toilet seat. Do not let collection paper touch the water. Pass bowel movement into collection paper.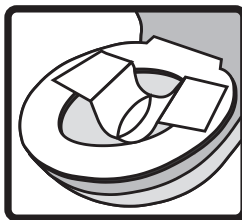


Step 4

Using the collection spoon built into the cap of the vial, collect small scoops of stool from each end and the middle. Replace cap and make sure the vial is tightly closed.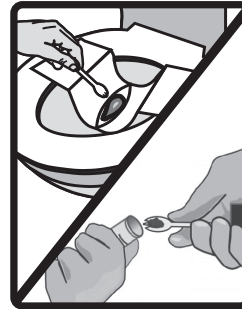

Step 5

Remove paper from toilet seat lid, drop paper in toilet bowl, wait a moment until paper is soft/wet and then flush.

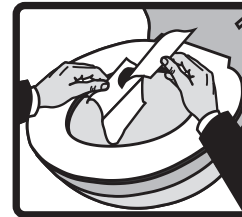
Step 6

Wash hands with soap and water.

Step 7

Put vial in plastic blue bag provided and refrigerate as soon as possible. Return the specimen to your physician or lab, preferably within 24 hours or no later than 48 hours of collection.

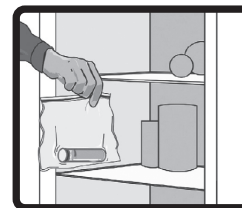Direct Testing of Stool Can Help Detect the Cause of Your Discomfort

Ulcers are sores in the surface of your stomach that often cause pain. H. pylori is a bacteria that is associated with about 80% of gastric ulcers and patients with H. pylori have 6X higher risk of developing gastric cancer^{1,2}. Antibiotic treatment can eliminate H. pylori and the risk of developing ulcers and the potential of gastric cancer.

Collecting a sample from your bowel movement is an accurate, easy, and fast way to find out if H. pylori is the source of your discomfort. Regardless of your H. pylori result, the information that your doctor receives will help treat your symptoms.

For technical assistance, please call Technical Services Department toll free at 1-844-TRY-HpSA (879-4772). To place an order call Customer Service Department at 1-800-543-1980. Both Departments are available M-F from 8AM to 6PM (Eastern Time).

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Reference:

1. Fennerty B. Helicobacter pylori: Why it still matters in 2005. Cleve Clin J Med 2005;72(2 Suppl):S1-S7.
2. Helicobacter and Cancer Collaborative Group. Gastric cancer and Helicobacter pylori: A combined analysis of 12 case control studies nested within prospective cohorts. Gut 2001; 49(3):347-353

Direct Detect[®] Collection System Sample Collection Instructions

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