

CONGENITAL CMV SALIVA

Sample Collection Guide

alethia™
CMV

1. Obtain a flocked swab - Flexible minitip flocked swab, plastic applicator, sterile, individually packaged.

For example:

- a. **Puritan** - HydraFlock® swab with or without viral transport media.

- Catalog Number (Peel Pouch): 25-3316-H
- Catalog Number (Dry Tube): 25-3316-H-BT
- Catalog Number (Swab in Conical Tube with 1 ml UTM): UT116

- b. **COPAN** - FLOQSwabs® with or without viral transport media.

- Catalog Number (Peel Pouch): 503CS01
- Catalog Number (Dry Tube): 553C
- Catalog Number (Swab in Conical Tube with 1 ml UTM): 360C

Or equivalent.



2. Confirm that the baby was breastfed > 1 hour prior to collection of the sample.

- a. For sample collection, baby could be held or in the bassinet.



3. Wash hands and/or put gloves on, then proceed to opening the sterile flocked swab.



- 4a. Introduce the swab between the cheek and the gum in one side of the mouth. Allow to sit for 10-15 seconds.



- 4b. Move the swab tip to the other side of the mouth for another 10-15 seconds. Make sure the swab appears moistened when removed.



5. Remove the swab from the mouth and replace in the holder (paper wrapper, plastic tube or tube with no more than 1mL UTM). The sample is stable 48 hrs at room temperature or 7 days refrigerated.
6. Proceed to transport the sample to the laboratory for testing with the Alethia™ CMV test.



This illustration is representative of the current Package Insert at the time of publication. Please refer to the most current version of the Package Insert for complete instructions.

USA/CCORPORATE OFFICE

3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513-271-3700

MERIDIAN BIOSCIENCE EUROPE

Belgium/Luxembourg Tel: +32 (0)6789 5959
France Tel: +33 (0)1 4256 0440
Germany Tel: +49 (0)3371 60 222 31
Italy Tel: +39 0331 43 3636
Netherlands Tel: +31 (0)411 62 11 66
United Kingdom Tel: +44 (0)20 8453 7970

Orders / Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Information Fax: 513-272-5432
Ordering Fax: 513-271-0124
meridianbioscience.com

meridian BIOSCIENCE™