IDkit Hp® Two

REF AC00050

EN Containing:
20 Baseline Breath Sample Bags
20 Post Ingestion Breath Sample Bags
20 Straws

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DE Inhalt:
20 'Baseline'-Atemprobenbeutel (Baseline Breath Sample Bag)
20 'Post Ingestion'-Atemprobenbeutel (Post Ingestion Breath Sample Bag)
20 Strohhalme

Meridian Bioscience Israel Ltd.
4 Ha'maayan St.
Modiin, Israel 7177872
Tel: +972-8-9737500
E-mail: modiin.contact@meridianbioscience.com

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EU REP

Donawa Lifescience
Piazza Albania, 10
00153 Rome, Italy

CH REP

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug, Switzerland

meridian BIOSCIENCE®

IDkit Hp® and BreathID® are trademarks of Meridian Bioscience Israel Ltd.

MLD00023 Rev. 05

EN | Breath Test for Detection of *H. pylori*

1 Instructions For Use

These Instructions for Use are intended to instruct how to use the included breath sample collection bags. The bags are used together with a ¹³C test meal (drink) that is supplied separately, to collect exhaled breath samples for the detection of *Helicobacter pylori*. Carefully read the Instructions for Use supplied with the test drink in addition to the following instructions.

Note: The instructions supplied with the test meal may describe using test tubes to collect the breath samples. Instead of test tubes, use the breath sample bags supplied with this kit, following the instructions below.

- The kit should be used by trained healthcare personnel.
- For use by adult patients and pediatric patients ages 3-17 years old.
- To be used with BreathID® Systems only.
- If the package or bags are damaged, DO NOT USE to avoid risk of patient contamination.
- For single use only! Do not attempt to sterilize bags for reuse to avoid risk of patient contamination or risk of sample carry-over.
- Please notify the Manufacturer/Distributor of any safety incident. In case of a serious incident also notify the local competent authority.

2 Required for each Individual Breath Test:

- One blue BASELINE Breath Sample Bag for BASELINE breath sample collection prior to ingestion of test substrate.
- One grey POST INGESTION Breath Sample Bag for POST INGESTION breath sample collection.
- One straw for mixing and drinking the test solution.

3 Direction for Use:

- 3.1 Identify the two Breath Sample Bags (the blue BASELINE bag and the grey POST INGESTION bag).
- 3.2 Prior to sampling the patient's breath, label each bag with barcode labels or write the necessary patient identification information in the appropriate fields on each bag.
- 3.3 Collection of the BASELINE breath sample:
 - 3.3.1 Remove the cap from the mouthpiece of the blue BASELINE bag. Keep the cap for closing the bag after collecting the breath sample.
 - 3.3.2 Instruct the patient to take a deep breath, hold their breath for 4 to 5 seconds and then exhale directly into the mouthpiece of the blue BASELINE breath bag until completely full.
 - 3.3.3 If the bag is not full, repeat step 3.3.2.
 - 3.3.4 Place the cap on the bag mouthpiece and firmly press until it clicks and is securely locked into place.
- 3.4 Prepare the test drink according to the test drink instructions.
- 3.5 Administer the prepared test drink to the patient. Ensure that the patient drinks the entire solution through a straw.
- 3.6 Start the timer for the number of minutes according to the test drink instructions.
- 3.7 After the patient finishes drinking the solution, record the present time plus (+) the number of minutes according to the test drink instructions in the Time to Fill field on the grey POST INGESTION bag.
- 3.8 Collection of the POST INGESTION breath sample:
 - 3.8.1 After the number of minutes according to the test drink instructions, remove the cap from the mouthpiece of the grey POST INGESTION bag. Keep the cap for closing the bag after collecting the breath sample.
 - 3.8.2 Instruct the patient to take a deep breath, hold their breath for 4 to 5 seconds and then exhale directly into the mouthpiece of the grey POST INGESTION bag until it is full.
 - 3.8.3 If the bag is not full, repeat step 3.8.2.
 - 3.8.4 Place the cap on the bag mouthpiece and firmly press until it clicks and is securely locked into place.

Note: If the patient has not held their breath for 4-5 seconds or does not fill the bag completely, there is a possibility that a test result will not be obtainable.

Note: The bag is not fully closed if the cap does not click into place. Not fully closing the bag may cause the breath sample to slowly leak out.

4 Breath Sample Analysis:

Store the breath samples between 15°-30°C. A BreathID® Smart device or BreathID® Hp Lab System should be used to measure the filled Breath Sample Bags within 14 days of collection. For more detailed information regarding the step-by-step procedure and device operation, refer to the BreathID® Smart or BreathID® Hp Lab System Operator's Manual.

Contacts:
Please contact your local distributor first,
or
E-mail: info@meridianbioscience.eu
Technical Support: MBE-TechService@meridianbioscience.eu
www.meridianbioscience.com/BreathID

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