

breathID[®] smart

TEST PROCEDURE

How to perform the test


H. pylori

1.



- Make sure instrument status is "Ready" before loading a batch.
- On the screen, a flashing cursor on Pair #1 indicates the first patient sample is ready to be scanned.
- Scan the barcode of the Blue Baseline Breath Sample Bag.
- On the top panel of the device, insert the Blue Baseline Breath Sample Bag into the upper port for Pair #1.
- If the matching LED is lit green, the bag was inserted correctly and the image of the bag on the screen will change to blue with the barcode number listed below.

2.

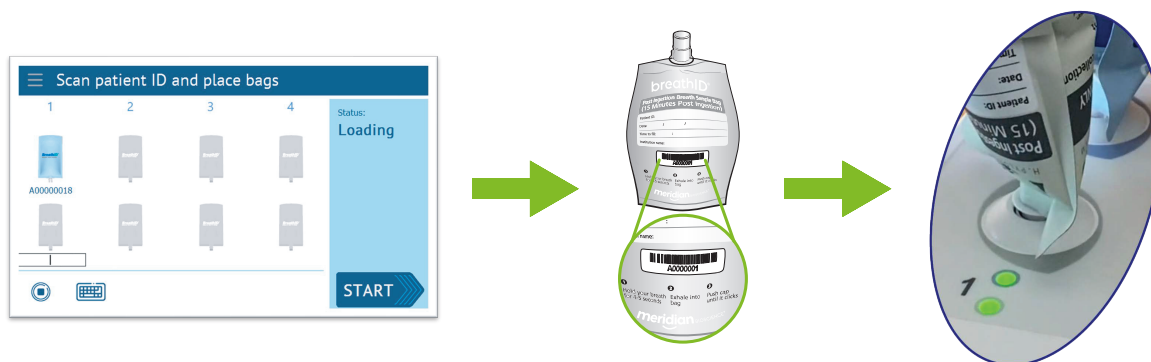

- Scan the barcode of the Gray Post Ingestion Breath Sample Bag.
- On the top panel of the device, insert the Gray Post Ingestion Breath Sample Bag into the lower port for Pair #1. If the matching LED is lit green, the bag was inserted correctly and the image of the bag on the screen will change to gray with the barcode number listed below.
- Ensure both LEDs are lit after properly inserting the bags in their corresponding sample ports on top panel of the device.

Note: To load more pairs of bags, repeat steps 1 and 2. A batch can be run with 1 to 4 tests loaded, but the pairs must be loaded in order, starting with Pair #1.



Read Operator's Manual

USA/CORPORATE OFFICE

3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513-271-3700

MLD01039 REV-01

MERIDIAN BIOSCIENCE EUROPE

Belgium/Luxembourg Tel: +32 (0) 6789 5959
France Tel: +33 (0) 1 4256 0440
Germany Tel: +49 (0)3371 60 222 31
Italy Tel: +39 0331 43 3636
Netherlands Tel: +31 (0) 411 62 11 66
United Kingdom Tel: +44 (0) 20 8453 7970

Orders/Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Information Fax: 513-272-5432
Ordering Fax: 513-271-0124
meridianbioscience.com

meridian BIOSCIENCE[®]


breathID[®] smart

TEST PROCEDURE

How to perform the test


H. pylori

3.

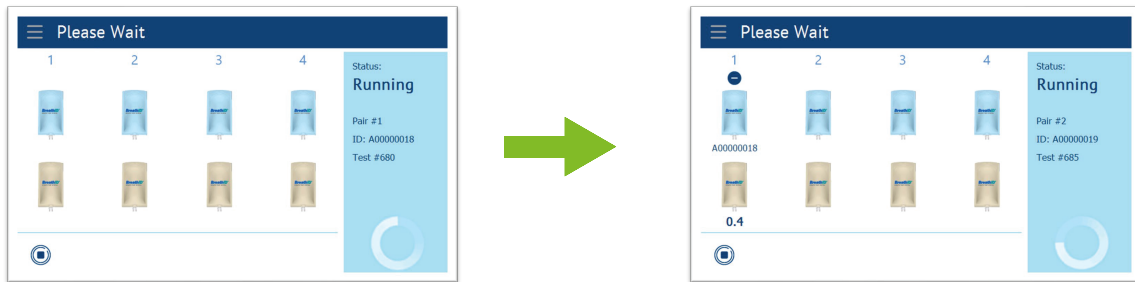

- Press Start
- The Status changes to "Running" and the barcodes become hidden. Pair #1 is analyzed first and an indicator spins until the batch is completed.
- After Pair #1 is analyzed, the result is displayed, and Pair #2 analysis begins. The remaining pairs of bags are analyzed the same way.

4.

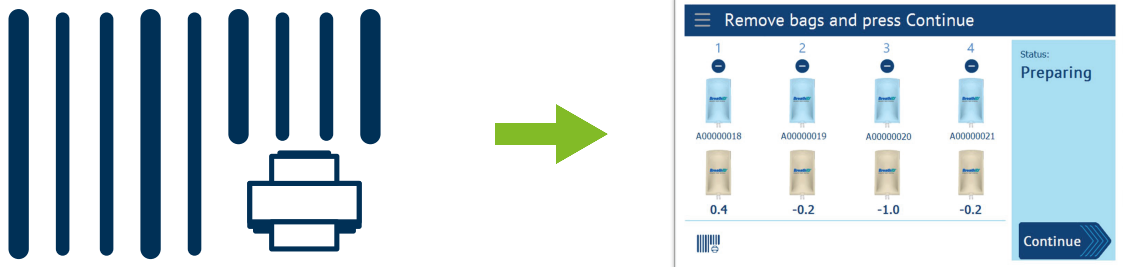

- To print control barcode labels, press the barcode printer icon at the bottom left corner of the screen before pressing Continue.
- Choose the tests you want to use as controls and place one label on each bag. Be sure the labels correspond with the correct test according to the pair number.
- Replace the bag caps and store for future use as controls.
- After the last pair has been analyzed the screen will display "Remove bags and press Continue". Once this is completed, the system will automatically prepare for the next batch.



Read Operator's Manual

USA/CORPORATE OFFICE

3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513-271-3700

MLD01039 REV-01

MERIDIAN BIOSCIENCE EUROPE

Belgium/Luxembourg Tel: +32 (0) 6789 5959
France Tel: +33 (0) 1 4256 0440
Germany Tel: +49 (0)3371 60 222 31
Italy Tel: +39 0331 43 3636
Netherlands Tel: +31 (0) 411 62 11 66
United Kingdom Tel: +44 (0) 20 8453 7970

Orders/Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Information Fax: 513-272-5432
Ordering Fax: 513-271-0124
meridianbioscience.com

meridian BIOSCIENCE[®]