breathID® smart

Н	D)	0	r



- Make sure instrument status is "Ready" before loading a batch.
- On the screen, a flashing cursor on Pair #1 indicates the first patient sample is ready to be scanned.
- Scan the barcode of the Blue Baseline Breath Sample Bag.

TEST PROCEDURE

1.

How to perform the test

- On the top panel of the device, insert the Blue Baseline Breath Sample Bag into the upper port for Pair #1.
- If the matching LED is lit green, the bag was inserted correctly and the image of the bag on the screen will change to blue with the barcode number listed below.



- Scan the barcode of the Gray Post Ingestion Breath Sample Bag.
- On the top panel of the device, insert the Gray Post Ingestion Breath Sample Bag into the lower port for Pair #1. If the matching LED is lit green, the bag was inserted correctly and the image of the bag on the screen will change to gray with the barcode number listed below.
- Ensure both LEDs are lit after properly inserting the bags in their corresponding sample ports on top panel of the device.

Note: To load more pairs of bags, repeat steps 1 and 2. A batch can be run with 1 to 4 tests loaded, but the pairs must be loaded in order, starting with Pair #1.



Read Operator's Manual

MANUFACTURER AND OUT OF USA DISTRIBUTOR: Meridian Bioscience Israel Ltd. 4 Ha'Maayan St. Modiin, Israel 7177872 Tel.: +972-8-973-7500

MLD01039 REV-02

USA DISTRIBUTOR: Meridian Bioscience Corporation 3471 River Hills Drive Cincinnati, OH 45244 Tel.: 1-800-543-1980

FOR BREATHID® SMART SYSTEM SERVICE INQUIRIES CONTACT: USA: modiin.support-USA@meridianbioscience.com

Out of USA: modiin.support@meridianbioscience.com

For more information regarding BreathID° Smart, visit our website at: www.meridianbioscience.com/breathid



breathID[®] smart





• Press Start

TEST PROCEDURE

How to perform the test

- The Status changes to "Running" and the barcodes become hidden. Pair #1 is analyzed first and an indicator spins until the batch is completed.
- After Pair #1 is analyzed, the result is displayed, and Pair #2 analysis begins. The remaining pairs of bags are analyzed the same way.



- To print control barcode labels, press the barcode printer icon at the bottom left corner of the screen before pressing Continue.
- Choose the tests you want to use as controls and place one label on each bag. Be sure the labels correspond with the correct test according to the pair number.
- Replace the bag caps and store for future use as controls.
- After the last pair has been analyzed the screen will display "Remove bags and press Continue". Once this is completed, the system will automatically prepare for the next batch.



Read Operator's Manual

MANUFACTURER AND OUT OF USA DISTRIBUTOR:

Meridian Bioscience Israel Ltd. 4 Ha'Maayan St. Modiin, Israel 7177872 Tel: +972-8-973-7500

MLD01039 REV-02

OF S Mariual

USA DISTRIBUTOR: Meridian Bioscience Corporation 3471 River Hills Drive Cincinnati, OH 45244 Tel: 1-800-543-1980

FOR BREATHID[®] SMART SYSTEM SERVICE INQUIRIES CONTACT: USA: modiin.support-USA@meridianbioscience.com Out of USA: modiin.support@meridianbioscience.com

For more information regarding BreathID® Smart, visit our website at: www.meridianbioscience.com/breathid

